Rollator RFM Standard, Silber

DEUTSCH

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Zimmer frame RFM Standard, Silver

ENGLISH

⚠ Carefully read the complete instructions for use before assembling, setting or using the Rehaforum Zimmer frame for the first time.

Intended Purpose: A Zimmer frame for the assistance with walking of people who are restricted in their movements and / or mobility. Seat function and tray, basket and shelf for the carrying of utensils and items of daily use.

Assembly / Instruction: Your specialist dealer or care worker will gladly help you with the assembly

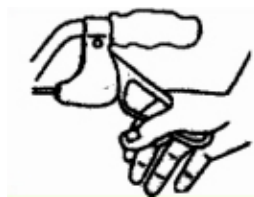
1. Remove all the parts from the packaging and place them within easy reach.
2. Unfold the Zimmer frame and allow the seat to click into place. To do this, pull the lever to the right on the rear of the frame. Now the rear leg supports can be pulled backwards out of the main body. Please ensure here that the lever on the rear locks into place. To fold the Zimmer frame back up again, the lever must be depressed. Thereby the locking mechanism is released.
3. Remove the caps on the upper ends of the Zimmer frame and fit the handles into the holders. The handles must be set parallel to the direction of movement.
4. Adjust the height to your requirements. Put the clamping screws into the clamp and ensure that the washer remains square on the bolt. Release the red button on the clamp fasteners so that they click into place. Then firmly tighten the nuts on the bolts.

Safety Instructions:

1. Ensure that the Zimmer frame is completely unfolded and that the seat is locked in place before you use the Zimmer frame.
2. Ensure that the handles are at the same height and test the stability of the Zimmer frame before you place your full weight on it.
3. Test the brakes before **every** use. When the parking brakes are engaged, the wheels must not turn, otherwise and the brakes / parking brakes must be readjusted.
4. Engage the parking brake before you sit on the Zimmer frame seat. Only sit when the Zimmer frame is standing on firm, level ground.
5. Replace visibly damaged or worn out wheels.
6. A Zimmer frame is no wheelchair but exclusively an aid to walking. Do not allow yourself to be pushed when sitting on the Zimmer frame seat.
7. Maximum user weight 130 kg.

Using the Brakes and Parking Brakes

1. Always use both the brakes and the parking brakes simultaneously.
2. In order to briefly slow down or stop the Zimmer frame, apply the brakes by simultaneously pulling the brake levers upwards until you have reached the desired speed or have come to a stop.
3. To lock the parking brakes, press the brake levers down simultaneously until they noticeably click into place.
4. To release the parking brakes, simultaneously pull the brake levers upwards until they are free of the lock.

Setting the Brakes before Use

The braking effect can be set directly on the brake lever (a) or on the Bowden cable (b). The brake is correctly set when the brake levers / Bowden cables have no play and with the brakes released the wheels rotate without making a scraping noise. (Specialist dealer)

Cleaning

The Zimmer frame should be cleaned and disinfected at regular intervals (absolutely necessary when changing user). Use a mild soap solution without scouring additives. The disinfection can be carried out using a commercially available agent in accordance with the manufacturer's instructions, for example, RFM Sept. Do not use abrasive or caustic cleaning agents, high pressure or steam cleaners!

Maintenance

The RFM Standard, Silver, Zimmer frame is maintenance-free.

Functional Faults:

If you discover a fading of the brakes, sluggishness of the wheels, irregular running of the wheels or other faults such as rust on the frame, please contact your specialist dealer.

Do not use a defective or damaged Zimmer frame!

Storage

Protect the Zimmer frame against moisture and keep it in a secure place to protect the wheels and frame against damage!

Contraindications

The use of the Zimmer frames is unsuitable in the event of

- Perceptual disorders
- Severe equilibrium disorders
- Loss of limbs on both arms
- Joint damage and / or joint contractions on both arms
- Reduced or inadequate eyesight

Scope of Supply

- RFM Standard Silver Zimmer frame including cane holder
- Shopping basket
- Tray
- Instructions for use

Technical Data / Dimensions and Weight:

Width: 60 cm

Seat width: 48 cm (seat surface: 38 cm)

Seat height: 61 cm

Seat depth: 20 cm

Handle height: 78.5 – 93 cm

Depth: 65 cm

Weight: 9 kg

Maximum loading capacity: 130 kg

Maximum load tray/basket: 4-5 kg

Guarantee

The guarantee period is 24 months and relates to all product defects which can be verifiably shown to be attributable to manufacturing or material defects.

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