

**Deluxe Swivel Cushion**  
**Cat. No.: 23513**  
**Instruction Manual**



The deluxe swivel cushion makes it easier to swing the legs and lower the body around. It facilitates turning into the right position for getting into and out of the car, and it's great on chair too. Ideal for

- Seniors,
- People recovering from surgery or injuries
- People with reduced lower body strength

**Features:**

- Secure. Non-slip base keeps cushion in place
- Added comfort ultra thick soft cushion
- Rotating 360 degree
- Hygienic washable cover easy to detach
- 100% polyester cover (hand wash)
- Diameter ca.39cm
- Maximum weight capacity:150kg

**How to use:**

- Place deluxe swivel cushion on the vehicle seat and lower yourself onto it.
- Holding on with your inside hand for support, bring your legs inside the vehicle, one at a time
- Holding with both hands get yourself to a comfortable position

## **CAUTION!**

Do not leave the deluxe swivel cushion on the floor – danger of sliding! Before setting off on your journey, check and make sure of the anti-slip qualities of the deluxe swivel cushion on your seat.

## **DISPOSAL:**

The packaging material is reusable. Dispose of the packaging in an environmentally friendly way by putting it in the appropriate recyclable waste collection containers.

Eurotops Versand GmbH  
Elisabeth-Selbert-Straße 3  
40764 Langenfeld, Germany



If technical support is needed, please call ...

in Germany            0180 . 530 63 63\* or [info@eurotops.de](mailto:info@eurotops.de)

in Austria             01 . 230 60 43 12 or [info@eurotops.at](mailto:info@eurotops.at)

in Switzerland       044 . 28 36 125 or [info@eurotops.ch](mailto:info@eurotops.ch)

in the Netherlands 026 . 37 36 333 or [info@eurotops.nl](mailto:info@eurotops.nl)

in France              0892 . 700 470\*\* or [info@eurotops.fr](mailto:info@eurotops.fr)

\* 14 Cents/minute via German landline, max. 42 cents/minute via mobile phone network

\*\*34 Cents/minute via French landline

A possible return of products should be addressed to the return address stated on your invoice.

The manual only reflects the technical conditions available under printing. Any changes in technology and equipment are reserved.

© 19.10.2013