GB/ US- digital Breath Alcohol Tester

1. LCD display, 2. switch, 3. Respiratory inhaler, 4.) Battery compartment
features

1. Breath alcohol test with audible warning
2. Measuring range 0.00 to 0.19 % BAC & 0.0-1, 9 g / l (or From 0.00 to 0.95 mg / l BRAC)
3. FAST response and result
4. Automatic shut off
5. operation with 2 x 1.5V AAA batteries (not included in delivery)

Inserting the batteries

1. Open the battery compartment by pushing down the cover.
2. Put in two 1.5V AAA batteries, be sure to the correct polarity as indicated inside the battery.
3. Close the battery compartment

Use of the breathalyzer

preparation

1. Press ON the power switch once, the display gets on and a beep sounds.
2. The warm-up will appear on the display icon and a countdown timer counts down from 10-0, the tester is in preparation mode.

hint

When first removed from the packaging or after a prolonged non-use of alcohol tester, are possible gases and air contaminations are on the respiratory inhaler, in which case the tester requires multiple preparatory phase, for this, please press the power switch more often.

use

1. The display shows the " Blow" symbol and a countdown counts from 10-0, the tester is ready for use.
2. Approach yourself with your mouth to the breath inhaler and blow for about 3-5 seconds into it.
3. Read the test result.
4. At reaching a value of 0.02 % BAC (oder 0, 2g / l) to 0.05 % (or 0.5 gll) also in the Caution symbol appears in the display :
0.02% BAC (or 0.2g / 1) to 0.05% BAC (or 0.5g / 1).
0 0.05% BAC (or 0.5g / 1) to 00:08 % BAC (or 0.8g / 1).
5. At reaching a value of 0.05% BAC (oder 0, 5g / l) or more, also the symbol Danger appears in the Display:
over 0.05% BAC & equal (or 0.5g / 1).
over 12:08 % BAC (or 0.8g / 1)

Automatic power off

1. The test result is displayed for about 15 seconds, then the . Off ' icon appears for 2- 3 seconds and the tester is shut off.
2. If during a test it is not blown into the Respiratory inhaler, the display shows 0.00 as Test result.

reference

In order to achieve a more accurate result, we recommend doing the test 2 or 3 times.

Exhausted battery indicator

If the . L symbol flashes when turned on the device, the batteries are exhausted batteries, please replace them.

Exchange the batteries at the latest when their capacity is not sufficient for switch on and prepare the device.

hand Strap

Please use the provided strap according to the following figure

specifications

- Sensor : Advanced semiconductor alcohol sensor oxygen
- Detection range 0.00 • 0.19 % BAC & 0.0 • 1.9 g / l (or From 0.00 to 0.95 mg / l BRAC)
- Accuracy: 0.01 % BAC (0.1 gll)
- Power supply: 2 x 1, 5V AAA batteries.
- Preparation time 10 seconds, the test result is < 5 seconds .
- Automatic switch off after about 35 seconds.
- Dimensions: 95 x 36 x 18mm.

precaution

- After drinking alcohol is recommended that at least 20 Wait for a while because it takes about 20 minutes to alcohol Is absorbed by the digestive organs in the blood and residual alcohol in the mouth needs to dissolve so long.

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