promed IRT-60

Infrarot-Fieberthermometer
Infrared clinical thermometer
Thermomètre infrarouge
Termometro a infrarossi
Termómetro infrarrojo para la fiebre
Infraroodkoortsthermometer
Инфракрасный медицинский термометр
Termometr medyczny na podczerwień
Infraröd febertermometer
Infrapunakuumemittari

DE

EN

FR

IT

ES

NL

RU

PL

SE

FI

DE

EN

FR

IT

ES

NL

RU

PL

SE

FI

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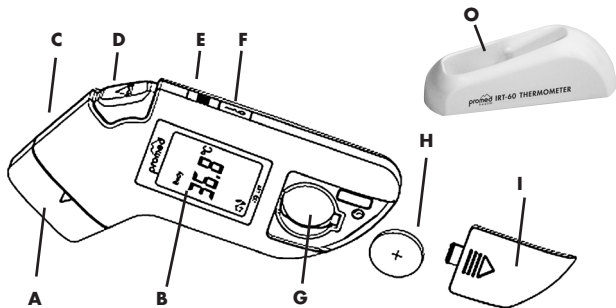
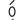
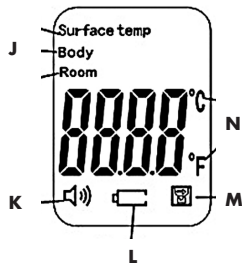
BRIEF DESCRIPTION OF THE DEVICE

- Non-contact infrared clinical thermometer
- Measurement of body, room and surface temperatures within seconds
- Hygienic and reliable measuring without skin contact
- 32 memory spaces
- Fever alarm
- Display of °C and °F
- Automatic switch-off (after 30 sec.)
- 3 different display colours for normal temperature, mild fever and fever

SCOPE OF DELIVERY/ACCESSORIES

- 1x Thermometer
- 1x Storage station
- 1x Button cell battery, 3V, type CR2032
- 1x Instructions for use

DESCRIPTION OF THE UNIT

Please keep this user manual in a safe place!

Please read the user manual thoroughly before using the device for the first time. Thank you very much for choosing the digital non-contact infrared clinical thermometer **Promed IRT-60**. In order to ensure the quality and safety of the device, this thermometer was designed and produced in compliance with the medical directive 93/42/EEC. We, as the manufacturer, cannot be made liable in any way for injury or damages to people or objects that arise from failure to comply with this user manual.

BEFORE USING THE THERMOMETER FOR THE FIRST TIME

1. Read these instructions of use with care and store them in a safe place.
2. Make sure that the thermometer was not damaged during transport.
3. Before using the thermometer for the first time, you must leave the **Promed IRT-60** in the room for 10 – 15 minutes in order for it to adjust to the room temperature. You can receive professional advice wherever **Promed** products are sold or get in touch with us if you have any questions. We can provide you with the name of the representative responsible for you.

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Further informations can be found on our homepage: www.promed.de

OPERATING THE UNIT

Prior to initial commissioning, please remove the contact protection strip from the battery. To do this, please proceed as described under "Battery change".

In order to switch on the thermometer, press the button **(F)**; Information illuminated in green is shown on the display **(B)**. This indicates that the thermometer is operable.

In order to carry out a measurement, hold the thermometer in such a way that the measuring sensor **(A)** is at a distance of 3 – 5cm to the surface to be measured and press the measuring button **(D)** once. You then hear a short beep and can read the measuring result from the display **(B)**. For each measurement, the indicator light **(C)** simultaneously lights up blue.

You can then switch off the thermometer by pressing the button **(F)** once. If you do not switch off the thermometer, it will switch off automatically after 30 seconds.

DE

EN

FR

IT

ES

NL

RU

PL

SE

FI

OPERATING THE UNIT

You can use the **Promed IRT-60** thermometer to measure body, room and surface temperatures.

Measuring the body temperature

What you should know before performing measurements:

The body temperature measurement delivers a measured value providing information about the current human body temperature. If you are uncertain about how to interpret the results or if abnormal values are measured, please contact your attending doctor. This also applies to minor changes in temperature if other symptoms (e.g. anxiety, excessive sweating, reddening of the skin, high pulse rate, tendency to collapse etc.) occur in addition. Temperatures measured with different thermometers cannot be compared to each other. For this reason, inform your doctor (or take into account in a self-diagnosis) of which thermometer you used and on which part of the body you carried out the measurement.

The reliability of the measuring result cannot be guaranteed if the temperature is measured in different parts of the body (e.g. first measurement on the forehead, second measurement on the arm).

Normal body temperatures related to the respective measuring location:

Body temperature range in degree (°) Celsius			
Designation	Measuring location		
	rectal	Oral / axillary	ear-thermometer
Subnormal temperature	- 36,2	- 35,8	- 35,2
Normal temperature	36,3 - 37,4	35,9 - 37	35,3 - 36,4
Increased temperature	37,5 - 38,0	37,1 - 37,5	36,5 - 37,0
Mild fever	38,1 - 38,5	37,6 - 38,0	37,1 - 37,5
Moderate fever	38,6 - 39,0	38,1 - 38,5	37,6 - 38,0
High fever	39,1 - 39,9	38,6 - 39,5	38,1 - 38,9
Very high fever	40,0 - 42,5	39,6 - 42,0	39,0 - 41,5
Death due to protein coagulation	ca. 42,6	ca. 42,1	ca. 41,6

In order to carry out a measurement, switch on the thermometer as described above and point the measuring sensor **(A)** to the forehead at a distance of 3 – 5 cm. Then press the measuring button **(D)** once. You hear a short beep and can read the measuring result from the display **(B)**.

With a measuring result below 37.3° C (99.1° F), the display **(B)** lights up green. This means that you have a normal body temperature.

With a measuring result between 37.4° C and 37.9° C (99.3° F and 100.2° F), the display **(B)** lights up orange. This means that you have an increased body temperature or mild fever.

With a measuring result above 38° C (100.4° F), the thermometer triggers an alarm. You hear a beep five (5) times and the display **(B)** lights up red. This means that you have a fever. If the temperature measured is below 32° C (89.6° F), the information "Lo" is shown on the display.

If the temperature measured is above 42.9° C (109.2° F), the information "Hi" is shown on the display.

OPERATING THE UNIT

Calibrating the thermometer

This function is only available in the "BODY" measuring mode. This function is designed to assist you in adjusting the **Promed IRT-60** to the measuring results of your usual clinical thermometer in order to achieve comparable measured values if you have always measured your body temperature using a thermometer rectally, but find this unpleasant. You can calibrate the **Promed IRT-60** by up to $\pm 3.0^{\circ}\text{C}$ ($\pm 5.4^{\circ}\text{F}$).

In order to perform the calibration, press the button **(F)** and keep it pressed for two (2) seconds. Then press the button **(F)** three (3) times.

You can then calibrate the thermometer as follows:

Pressing the measuring button **(D)** increases the value by 0.1°C .

Pressing the memory button **(E)** reduces the value by 0.1°C .

In order to save the set values, press the button **(F)** again.

Measuring the room temperature

In order to measure the room temperature, you must switch the thermometer to the "Room" measuring mode. To do this, press the button **(F)** and keep it pressed for two (2) seconds. Press then the button **(F)** another two (2) times.

By pressing the measuring button **(D)** or by pressing the memory button **(E)**, you can switch the measuring mode:

In order to save the set measuring mode, press the button **(F)** another two (2) times.

In order to perform a measurement, switch on the thermometer as described above. Then press the measuring button **(D)** once. You hear a short beep and can read the measuring result from the display **(B)**.

Please note:

The thermometer must have been in the room for at least 10 minutes before the measurement can be carried out.

Measuring surface temperatures

In order to measure the surface temperature, you must switch the thermometer to the "Surface temp" measuring mode. For this purpose, press the button **(F)** and keep it pressed for two (2) seconds. Press then the button **(F)** another two (2) times.

By pressing the measuring button **(D)** or by pressing the memory button **(E)**, you can switch the measuring mode:

In order to save the set measuring mode, press the button **(F)** another two times.

In order to perform a measurement, switch on the thermometer as described above and point the measuring sensor **(A)** to the surface to be measured at a distance of 3 – 5 cm. Then press the measuring button **(D)** once. You hear a short beep and can read the measuring result from the display **(B)**.

DE

EN

FR

IT

ES

NL

RU

PL

SE

FI

BATTERY CHANGE

If the battery power is too low, the symbol **(L)** appears on the display **(B)**.

In order to change the battery, remove the battery compartment cover **(I)** from the battery compartment **(G)** and remove the old battery **(H)**. You can then insert the new battery into the battery compartment **(G)** and slide the battery compartment **(I)** back onto the thermometer.

SWITCHING THE UNIT OF MEASUREMENT

In order to switch the thermometer's unit of measurement (Celsius / Fahrenheit), you must press the button **(F)** and keep it pressed for two (2) seconds. By pressing the measuring button **(D)**, you can switch the unit of measurement. In order to save the set unit of measurement, press the button **(F)** another four (4) times.

CALLING AND DELETING THE MEASURED VALUES SAVED

In order call measured values saved, you must first press the button **(F)** and then the memory button **(E)**. You can then read the last measured value from the display **(B)**. Every time the memory button **(E)** is pressed, you call the preceding measured value. The **Promed IRT-60** thermometer saves the last 32 measured values.

In order to delete the measured values, you must first press the button **(F)** and keep it pressed for four (4) seconds and then repeat this using the memory button **(E)**. As soon as a beep sounds, all measured values are deleted. In order to check whether the deletion was successful, press the memory button **(E)** once more and the information "CLr" is shown on the display **(B)**.

SWITCHING ON AND OFF THE SIGNAL SOUND

In order to switch on or off the signal sound, press the button **(F)** and keep it pressed for two (2) seconds. Then press the button **(F)** again.

In order to switch on the signal sound, press the measuring button **(D)** and the information "ON" is shown on the display **(B)**. In order to switch off the signal sound, press the memory button **(E)** and the information "OFF" is shown on the display **(B)**. In order to save the setting, press the button **(F)** another three (3) times.

CARE / MAINTENANCE / STORAGE

The **promed** thermometer **IRT-60** is water-proof and therefore easy to clean and disinfect. The measuring sensor **(A)** is the most sensitive part of the thermometer. Only use a damp cloth or a cloth soaked in mild alcohol solution in order to clean it. Never immerse the thermometer in water. Remove the battery from the thermometer when not using it for a longer period of time. Never expose the thermometer to direct sunlight. Make sure that the thermometer does not fall down.

SAFETY INSTRUCTIONS

-  Never deposit or use the thermometer on hot surfaces, close to open gas flames or hot objects such as stove tops. Otherwise, the housing will melt.
- Use the thermometer only with batteries approved by the manufacturer.
- Use the thermometer indoors only.
- Keep the thermometer in a dry place.
- Some kind of supervision is required when using the thermometer around children.
- Children must not play with the thermometer. It includes small parts (battery and battery cover) which may cause suffocation if swallowed.
- Use the thermometer only for applications as specified in these instructions for use.
- The thermometer is exclusively intended for measuring human body temperature, as well as for the measuring spots in the human body as specified in the instructions for use.
- Using the thermometer in the vicinity of strong electromagnetic fields, e.g. next to a mobile telephone, may cause malfunctions.
- Please immediately contact your general practitioner in case of conspicuous results.
- This thermometer is not intended for use by persons (including children) with restricted physical, sensorial or mental abilities or lacking experience and/or lacking knowledge, unless they are supervised by a person responsible for their safety or if they were instructed on the use of the thermometer.
- Never allow children to play with the packing material, as there is a danger of suffocation.
- Cleaning and general maintenance work may only be carried out by children when supervised.

METROLOGICAL CHECK (FOR COMMERCIAL USE)

The thermometer was calibrated by the manufacturer for the duration of two years. For commercial use, a metrological check must be made at least every two years. The check is subject to a charge and can be accomplished by the responsible authorities or by authorised maintenance service agencies in compliance with the "Medical Device Operator Ordinance".

DISPOSAL

Power tools, accessories, and packaging should be recycled in an environmentally suitable manner.

Only for EU countries:

Do not throw power tools in with the domestic waste! In accordance with the European Directive 2002/96/EC on waste electrical and electronic equipment and its implementation into national law, nonfunctioning power tools must be collected separately and recycled in an environmentally suitable manner.

This symbol indicates that disposal of this product in the domestic waste is not permitted within the EU. Waste devices contain valuable recyclable material

DE

EN

FR

IT

ES

NL

RU

PL

SE

FI

DISPOSAL

that should be recycled. Recycling also prevents uncontrolled waste disposal from damaging the environment and human health. Therefore, please dispose of all waste devices using the appropriate collection systems or send the device to the place of purchase for disposal. The place of purchase will then recycle the device.

Rechargeable batteries/batteries:

Do not throw your rechargeable batteries/batteries in the domestic waste, in fire, or in water. Rechargeable batteries/batteries should be collected, recycled, or disposed of in an environmentally friendly manner.

Only for EU countries:

According to the directive 91/157/EEC, defect or used rechargeable batteries/batteries must be recycled. Waste rechargeable batteries/batteries can be directly handed to:
Promed GmbH, Lindenweg 11, D-82490 Farchant.

TROUBLESHOOTING

If you have problems while using the thermometer, solutions for the most common causes of errors can be found below. If the problem persists, please contact our customer service.

Problem	Possible cause	Possible remedy
When performing a body temperature measurement, measured values below 32° C (89.6° F) are shown on the display (B) .	The thermometer has not been set to the measuring mode "Body".	Change the measuring mode to "Body" according to the instructions for use.
When carrying out the measurement, "Hi" is shown on the display (B) .	The measured temperature exceeds the maximum thermometer's measuring range.	Check the selected measuring mode and change or adjust it if necessary. If the measuring mode has been set correctly, the object to be measured may be too hot.
When performing the measurement, "Lo" is shown on the display (B) .	The measurement is impaired by hair or sweat.	Make sure that the measurement is not impaired by any obstacles or moisture.
	The measurement is impaired by a draught or a significant change in the ambient temperature.	Make sure that no draught or significant change in temperature occurs while carrying out the measurement.
	The measurements have been carried out within too short an interval and the thermometer did not have enough time to restart.	Leave at least three (3) to five (5) seconds between the measurements; a 15-second interval is recommended.
	The measuring distance is too large.	Repeat the measurements with the recommended distance (approx. 3 - 5 cm, 1.2 to 2.0 in).

TROUBLESHOOTING

When performing measurements within a short period of time, the results vary greatly.	The measurement was not carried out correctly or without complying with the required intervals.	Repeat the measurement correctly.
	The battery is dead.	Replace the battery.
The battery symbol (L) lights up on the display (B) .	The battery is dead.	Replace the battery.
The thermometer does not carry out measurements, but it can be switched on.	The measuring sensor (A) may be dirty.	Clean the measuring sensor (A) according to the instructions above.

WARRANTY & SERVICING

This product was produced with the utmost care and was thoroughly tested before it left the factory. Therefore, we provide a warranty period of 24 months from the date of purchase for this product. Promed products conform to their description and the relevant specifications; it is your responsibility to ensure that the product which you have purchased is designed for your intended use.

- In the event of material defects and/or manufacturing errors which appear during proper use and are detected during the warranty period, we will replace all defective parts of the product free of charge within the warranty period, including the labour costs for warranty repairs.
- The warranty does not cover:
 - Normal wear on the product
 - Defects which were caused during transportation or storage of the product
 - Defects or damage which were caused through improper use or inadequate maintenance
 - Damage resulting from disregarding the notes in the operating manual
 - Damage resulting from modifications to the product which were not carried out by Promed.
 - Damage caused by sharp objects or as a result of torsion, compression, being dropped, an abnormal impact or other actions which are beyond Promed's reasonable control.
 - Parts which are subject to wear (e.g. moving parts such as ball bearings, etc., fasteners, etc.) are generally excluded from the warranty.
- A claim under the warranty is not enforceable if:
 - the product is not sent back in its original packaging or in adequately secure packaging;
 - it has been modified or repaired by a person or company other than Promed or one of Promed's authorised dealers;
 - the product has been repaired using parts which are not approved by Promed;
 - the product's serial number has been removed, erased, modified or made unreadable;
- For reasons of hygiene, any products which have been exposed to direct contact with a body or with bodily fluids (e.g. blood) must be packaged in an additional plastic bag before they are sent back. In this case, a specific note of this situation must be made in the package or in the accompanying documents.
- For components or products which are replaced during the warranty period, a warranty is only provided for the remainder of the original warranty period; provided that the product or part replacement was carried out by Promed or one of Promed's authorised dealers.

DE

EN

FR

IT

ES

NL

RU

PL

SE

FI

WARRANTY & SERVICING

- The warranty period begins on the day of purchase. Claims under the warranty must be established within the warranty period. Complaints arising after the expiry of the warranty period cannot be considered.
- Under these warranty conditions, the warranty shall only apply if the date of purchase can be proven by means of a sales receipt or similar.
- We reserve the right to make technical and visual changes as well as changes to the features.
- This warranty is only valid and enforceable in the country in which it was purchased by the original buyer, provided that it was Promed's intention that the product should be offered for sale in this country. This warranty is also enforceable in any country within the European Economic Area in which Promed has an authorised importer or sales partner. Depending on each country, specific and differing warranties and guarantees may be applicable owing to the particular applicable legislation. These legal provisions are neither excluded nor limited by these warranty conditions. To the extent permissible under national law, the warranty period shall not be extended, renewed or otherwise affected by any subsequent resale, repair or replacement of the product.
- The provisions of the UN Convention on Contracts for the International Sale of Goods shall not apply.
- Our warranty conditions do not affect the seller's legal warranty obligation.
- To the fullest extent possible under the applicable mandatory legislation, these warranty conditions constitute your sole and exclusive legal remedy and shall apply in place of any other express or implied warranty conditions. Promed is not liable for any unusual, incidental, punitive or consequential damages including, but not limited to loss of profits, loss of use, loss of revenue, costs of replacement equipment or facilities, third party insurance claims, and damage to property which are the result of the purchase or use of the product or occur owing to a breach of warranty, a breach of contract, negligence, product defects or any other legal or statutory circumstances, even though Promed was aware of the possibility of such damages. Promed is not liable for any delay in warranty claims.
- Promed cannot be held liable for any possible translation errors.

The following information is essential for smooth processing:

1. Original proof of purchase/receipt or dealer's stamp with date of purchase
2. Defect discovered
3. Product designation / type / serial/batch number

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Promed GmbH, Lindenweg 11, D-82490 Farchant
Model Name: IRT-60
Voltage: 3V^{CR} (CR 3032)
Operating Current: 150 mW
Measuring Distance: 3-5cm
Promed-No.: 470030
Lot: 
 Guangzhou Jinxinbao Electronic Co.,Ltd;

ZERTIFIZIERUNGS-NR | CERTIFICATION No | N° DE CERTIFICATION | NR. DI CERTIFICAZIONE |
N.º CERTIFICADO | CERTIFICERINGSNR | СЕРТИФИКАЦИОННЫЙ НОМЕР | NR CERTIFIKATU |
CERTIFIERINGS-Nr | CERTIFIKOINTINUMERO

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Symbol for WARNING! Indicates sections of the user manual that are safety-relevant!

The symbol corresponds to symbol 10, table D1, annex D of the EN 60601-2006+AC:2010 Standard Instructions for a unit, system or accessories contain important safety-relevant information for the consumer.

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Symbol for type BF (Body Floating) devices

The symbol corresponds to symbol 2, table D2, appendix D to standard EN 60601-2006+AC:2010. Devices complying with the requirements of this standard offer a higher level of protection against electric shock, especially regarding permitted patient leakage currents and user safety.

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Follow the instructions for use!

The symbol corresponds to symbol 10, table D2, appendix D of standard EN 60601-1:2006+AC:2010. The information provided in the instructions for use must be complied with.

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Symbol for Manufacturer and manufacturing factory

The symbol corresponds to symbol 5.1.1 of the EN 15223-1:2012. Standard. It shows the name and the address of the manufacturer and the manufacturing factory.

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Symbol for date of manufacture

The symbol corresponds to symbol 5.6 of the EN 15223-1:2012. Standard. It shows the date of manufacture.

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Symbol for Notified Body

This Symbol Complies with the European Medical Device Directive 93/42/EEC and amended by directive 2007/47/EC requirements. Notified body TÜV Süd (Technical Control Board, South) (0123).

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Symbol for Authorized representative in the European Community

The symbol corresponds to symbol 5.1.2 of the EN 15223-1:2012. Standard. It shows the Name of Authorized representative in the European Community.

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Guangzhou Jinxinbao Electronic Co., Ltd
Wellkang Ltd; SuiteB, 29 Harley Street, London