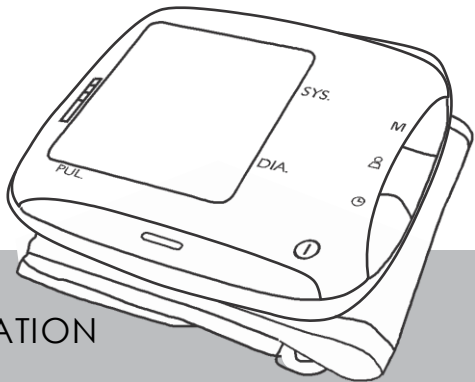
TENSIOMETRE POIGNET EXTRA PLAT

FR

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REF : MD2200 – T 2020

CONSIGNES DE SECURITE

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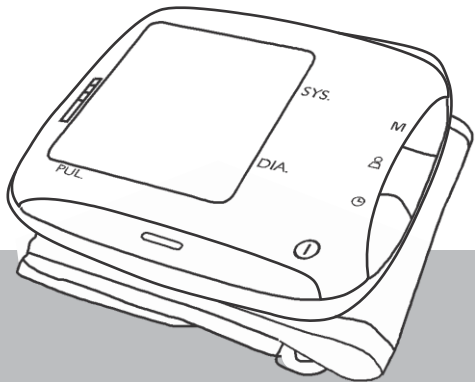
CAPTELEC - FRANCE
59 170 – Croix

DIGITAL AUTOMATIC
WRIST BLOOD PRESSURE MONITOR

EN

USER MANUAL

REF : MD2200 – T 2020

SAFETY NOTES

CAUTION

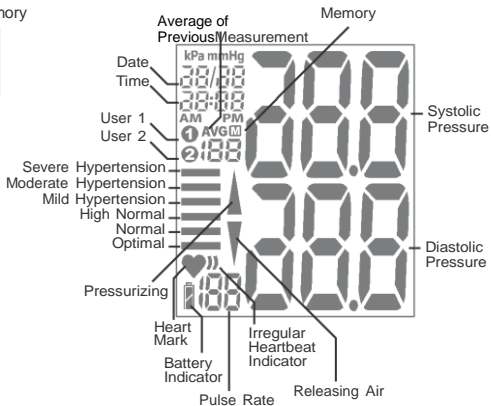
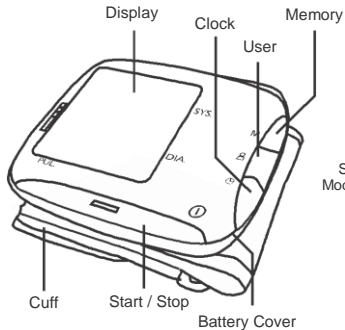
- This item is exclusively dedicated for a household use.
- If you have any medical doubt, report it to your physician before using the wrist blood pressure.
- Use the wrist blood pressure only as precised in the instruction manual. Any other use cancels the warranty rights.
- If you have some health problems, for example an arterial occlusive disease, ask to your physician before use.
- This item should not be used for pacemaker measurement
- Pregnant women must take care and take into account their personal limits. If needed, please consult your physician.
- If you feel, during measurement, an embarrassment, for example a pain in the arm, deflate the cuff immediately. Undo the cuff and remove it.
- This instrument is not for person (including children) with physical capacity, sensorial or mental reduced or without experiences and/or knowledge, unless they are supervised from a person taking care of they security or they have been taught about the utilization of the instrument.
- Children must be supervised paying attention to the fact that they are not playing with the instrument.
- The instrument may not be used near another instrument from which some important electric radiations are issues, for example a radio, cellphones o microwaves.
- If something is happening on the instrument don't fix it yourself, it can cancelled the warranty. Please contact an authorized center to fix it up.
- Keep the instrument away from humidity. If the instrument is in contact with humidity, take off the batteries and don't use it anymore. Please contact the seller or get informed directly.
- If you are not using the instrument in a constant way, take off the batteries.

WARNING


Don't take any therapeutic caution on your one analysis. Never change the medication doses prescribed by your doctor.

PART IDENTIFICATION & DISPLAY READINGS

INSTALL/ CHANGE BATTERIES

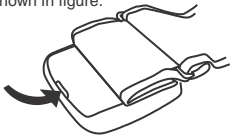1. Pull the battery cover located on the bottom side on the unit as shown in figure.

2. Remove old batteries and insert new ones. Use LR03 or AAA alkaline batteries and use batteries of same type only.
3. Make sure the battery polarities (+) and (-) match the marking on the battery compartment.

4. Place back the battery door.

NOTE:

- Insert the batteries as shown in the battery compartment. If not, the device will not work or even be damaged.
- [] icon represent battery power is enough.
- When battery power becomes weak, the [] icon and "E6" appears in the display, replace all batteries with new ones. Do not mix old and new batteries.
- Never use rechargeable batteries. This may damage the unit.
- Battery life may vary with ambient temperature and may be shorter at low temperature.

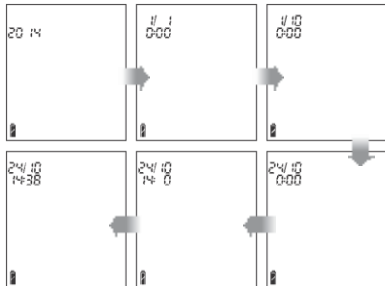
TO SET CLOCK

When new batteries are installed

1. "YEAR" will blink on display automatically.
2. Press [+ / M] to select current Year.
3. Press [⏻] button to confirm and "MONTH" will display.
4. Press [+ / M] to select current Month.
5. Press [⏻] button to confirm and "DAY" will display.
6. Press [+ / M] to select current Day.
7. Press [⏻] button to confirm and "HOUR" will display.
8. Press [+ / M] to select current Hour.
9. Press [⏻] button to confirm and "MINUTE" will display.
10. Press [+ / M] to select current Minute.
11. Press [⏻] button to confirm and clock setting is done.
12. The set date and time will be displayed.

At any time

1. [⏻] button to when the device is off.
2. Date and time will be displayed.
3. Press and hold [⏻] button for 3 seconds and "YEAR" will blink.
4. Follow the same procedure above to set clock.

TO APPLY WRIST CUFF

1. Bare your left wrist, make sure that the circulation in the arm is not restricted by any cloth etc. that are too tight. Position the cuff on the inside of your wrist.

2. Fasten the cuff with the Velcro fastening so that the upper edge of the monitor is positioned approximate 1 cm below the ball of your thumb.

3. The cuff has to be fitted tightly around the wrist but should not constrict it.

BEFORE MEASURING BLOOD PRESSURE

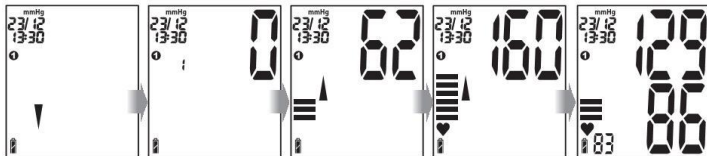
1. Apply the wrist cuff following the instruction in “TO APPLY WRIST CUFF”.
2. Rest at least for 5 minutes before each measurement. Otherwise there may be erroneous results.
3. Sit down in a comfortable position, which your legs should be uncrossed, feet should be feet on the floor and your back should be supported by a chair. At the same time your wrist should be supported on a flat surface which the cuff is on a level with your heart.
4. Relax your wrist and turn your palm upward.
5. Relax, keep still and do not talk during the measurement in order not to distort the result.

NOTE:

- This instrument is intended for use by adult only. Do not use this device on or by children, toddlers and infants.
- For reliable monitoring and reference of blood pressure, it is recommended to do the measurement daily at the same time.
- In order to obtain a resting condition blood pressure, do not eat, drink alcohol and caffeinated beverages, smoke, exercise or bath for at least 30 minutes before taking a measurement since your blood pressure varies time to time depending on what you have eaten, drunk and what you have done. Stress also raises blood pressure.
- You should not be physically tired or exhausted while taking measurement.
- Perform measurements in a quiet and relaxed environment at room temperature.
- Remain relax, still and do not speak during measurement since the accuracy of any blood pressure measurement with this device can be affected by those multitude of causes.
- Always wait at least 5 minutes between measurements to allow the blood circulation in your wrist to return to normal. You may need to increase the wait time depending on your individual physiological characteristic.
- Should the device detect an abnormal condition, it stops the measurement and display an error code. See “TROUBLESHOOTING” for more details.
- This device bases it measurements on your heartbeat. If you have a very weak or an irregular heartbeat, the device may have difficulty in determining your blood pressure. (An irregular heartbeat is defined as a heartbeat that varies by 25% from the average of all heartbeats during measurement.)

TO MEASURE YOUR BLOOD PRESSURE

1. Follow the instruction in "BEFORE MEASURING BLOOD PRESSURE" to make yourself ready.
2. Press [①] button when the device is off to start measurement. The average blood pressures and last data number are displayed briefly. The current pressure is displayed, as the measurement begins.
3. The cuff starts to inflate. It is normal for the cuff to feel very tight. A pressure bar indicator is displayed during measurement. See "PRESSURE BAR INDICATOR" for more details.
4. When inflation is complete, deflation starts automatically. Once the pulse is detected, the [♥] blinks with each pulse beat, indicating that the measurement is in progress.
5. When the measurement is complete, the systolic and diastolic pressure and pulse rate are displayed and stored. At the same time, the cuff exhausts the remaining air and deflates completely.
6. Press [①] button to turn off the power.

Deflation for start
measuring

Measurement record
number shown

Inflation in progress

Measurement
in progress

Measurement
result shown

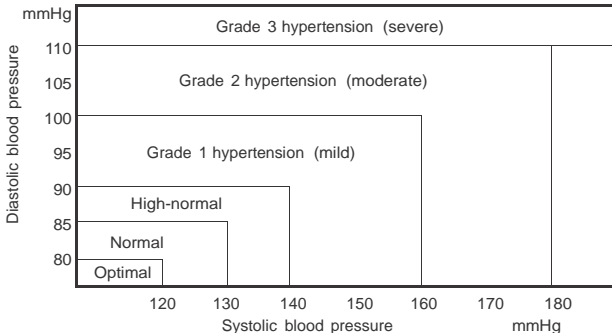
Note:

You can stop inflation by pressing [①] button at any time.

PRESSURE BAR INDICATOR

Standards to assess high blood pressure, without regard to age, have been established by the World Health Organization (WHO), as shown in the chart below.


Reference Material: *Journal of Hypertension* 1999, Vol 17 No.2


WHO CLASSIFICATION INDICATOR

Each of the six segments of the bar indicator corresponds to the WHO blood pressure classification.

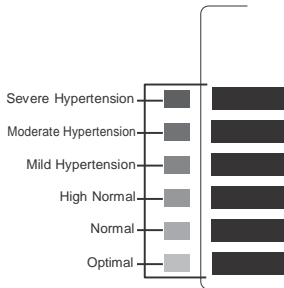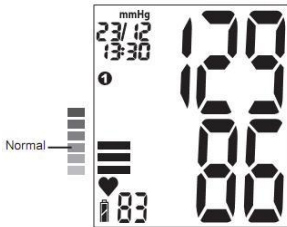WHO Classification Indicator:

Example:

The indicator displays a segment, based on the current data, corresponding to the WHO classification.

TO RECALL AVERAGE AND PREVIOUS MEASUREMENT DATA

This device has a memory capability to store the measurement readings for each user. Every time you complete the measurement, the device automatically stores the measurement result.

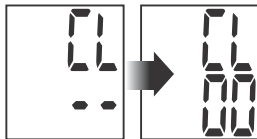
1. To get the average measurement data of the last 3 measurements, press [] key to turn on the unit.
2. Press [] key to select the user you desire.
3. Press [+/M] to read the average data.
4. Press [+/M] continuously to view previous measurement data.
5. Press [] button to turn off the device.

TO DELETE MEASUREMENT DATA

This monitor has 2 methods to delete the recorded memory.


(A) To delete one single record

1. Select the record you want to delete
2. Press and hold [+/M] button until the screen display "CL" and "--"
3. Press [$\frac{0}{\Delta}$] button to confirm and the screen will display "CL" and "00"

(B) To delete all memories

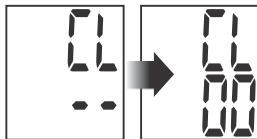1. Press the [+/M] button to enter average mode
2. Press and hold [+/M] button until the screen display "CL" and "--"
3. Press [$\frac{0}{\Delta}$] button to confirm and the screen will display "CL" and "00"

WHAT IS AN IRREGULAR HEARTBEAT

This blood pressure monitor provides a blood pressure and pulse rate measurement even when an irregular heartbeat occurs. An irregular heartbeat is defined as a heartbeat that varies by 25% from the average of all heartbeats during the blood pressure measurement. It is important that you are relaxed, remain still and do not talk during measurements.

Note: We recommend contacting your physician if you see this [♥👉] indicator frequently.

TROUBLESHOOTING

Nothing appears in the display, even when the power is turned on	Batteries are drained	Replace all batteries with new ones
	Battery polarities are not in the correct position	Re-install the batteries with their negative and positive ends matching their indicated in the battery compartment
ERROR code 1 (E1) appears	The cuff position is not fastened properly	Fasten the cuff correctly
	The cuff position is not correct	Sit comfortably and still. Ensure that the cuff is the same level as the heart
ERROR code 2 (E2) appears	You moved your arm or body during measurement	Make sure you remain very still and quiet during the measurement

TROUBLESHOOTING

ERROR code 3 (E3) appears	The cuff may not be applied	Check whether tube connection of the cuff is secured to the unit properly
ERROR code 4 (E4) appears	The unit does not measure	If you have a very weak or irregular heart beat, the device may have difficulty in determining your blood pressure
	There is a measuring Error	Sit comfortably and still. Fasten the cuff again carefully
ERROR code 5 (E5) appears	Cuff over inflated	The measurement range is over 300 mmHg. It is recommended to see doctor as soon as possible.
ERROR code 6 (E6) appears	Low battery	The battery power is too low to function. Replace the batteries with new ones.
The monitor keeps reinflating	Circuit locked	Remove and reinsert the batteries and then proceed to take measurement again.

TECHNICAL SPECIFICATION

Display	: LCD Display
Measurement Range	: Systolic Pressure: 50-250 mmHg ; Diastolic Pressure: 30-200 mmHg Pulse : 40-180 beats/minute
Accuracy	: Pressure : +/-3 mmHg Pulse : +/-5% of reading
Resolution	: Pressure : 1 mmHg Pulse : 1 beat / minute
Measurement Method	: Non-invasive, Oscillometric method
Power Source	: 2 x 1.5V AAA batteries
Operating Temperature / Humidity	: +5°C to +40°C, 15-93% RH maximum
Storage Temperature / Humidity	: -25°C to +70°C, up to 93% RH maximum
Operation, storage and transport atmospheric pressure	: 700hPa to 1060hPa
Outer Dimensions	: Approx. 84 x 77 x 15mm
Wrist Circumference	: 13.5 - 23cm
Accessories	: Cuff, User manual, Storage box, Batteries(Optional)
Classification	: Application part Type BF
Key to symbols	: Application part Type BF  Class II equipment symbol 

The European directive 2012/19/EC about electronic and electric waste, requires that you can't throw away defective domestic appliance with common waste. Used device has to be collect separately to optimize the recuperation rate and the recycling of the materials to reduce the impact on the health and the environment.

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capacities, or by persons lacking in experience or knowledge, unless they are supervised or instructed on the use of this apparatus by a person responsible for their safety. Children should be supervised to ensure they do not play with this appliance.

Precision and precautions about the battery use

- Throwing away a used battery in the environment or with household waste pollutes and avoids the recuperation of reusable materials.
- It is important to limit the battery consumption and to respect the following rules:
 - Favor alkaline battery (last longer than saline battery) and when it's possible use rechargeable battery.
 - Leave used batteries and accumulators in adapted containers that you can find in some retailers. In that way, metals don't pollute the environment and the health.

- Battery has to be inserted by respecting the polarity show on the apparel and the battery. An incorrect positioning can damage the apparel, cause leaking and even provoke fire and the explosion of the battery.
- To assure a good functioning, battery has to be in a good condition. If there is a problem, put new battery.
- Never try to recharge battery not made for this. It could leak, warm up, cause fire or explode.
- Replace all the battery at the same time. Never mix up alkaline battery with saline battery.
- Used battery has to be removed.
- Also, remove the battery if you're not using the apparel for a long time.
- Never try to short-circuit the terminal of a battery.
- Never throw away the battery in fire because they can explode.
- Take out the accumulator from the apparel before reload it.
- We recommend that an adult watch if a child change the battery to respect those rules.
- If you swallow a battery, go immediately to your doctor or the hospital.
- For the best ongoing performance, we recommend using only fresh, high-quality, alkaline batteries.

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