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

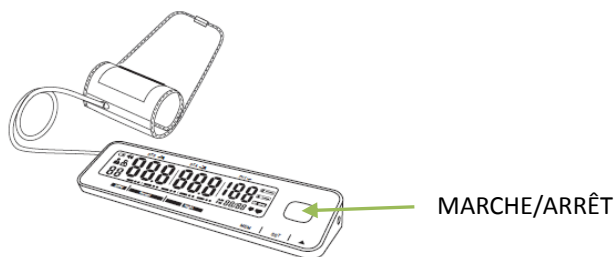
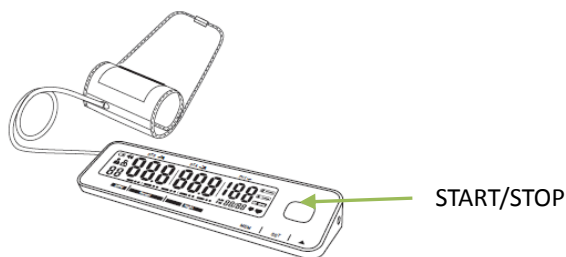
Warning: Please read this instruction manual carefully before using the instrument and keep it for future use.

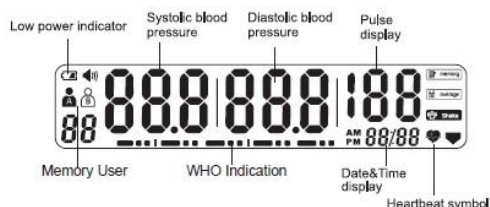1. Display

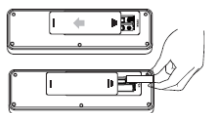2. Battery Replacement

- Open battery cover at the back of the monitor.
- Load four "AAA" size batteries. Please pay attention to polarity.
- Close the battery cover.

3. Setting the date and time

To settle the date and time, turn OFF the device. Press the setting button SET for a while. The year flashes on the display. Press Δ to advance by increments of one year and to set the current year.

The year can be set between 2010 and 2030. When the display reaches 2030, it will return to 2010.

Press SET again, the date flashes on the display. Press Δ to set the date.

Then press SET again, the month flashes on the display. Press Δ to set the month.

Then press SET again, the hour flashes on the display. Press Δ to set the hour. Then press SET again, the minutes flash on the display. Press Δ to set the minutes. Press SET when finished to register the settings and back to normal screen.

Press and hold the button Δ to increase the date and time values faster.

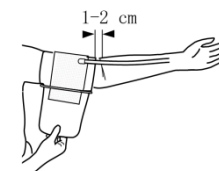
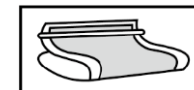
Press the START/STOP button to turn the monitor off.

4. Set user

To choose user A or B before to measure your pressure, turn OFF the device using the START/STOP button. Hold the START/STOP button and check the left upper part of the display screen : user A or user B will flash. Then release the button and start using your device.

5. Applying the cuff

- Pulling the cuff end through the medial loop (the cuff is packaged like this already), turn it outward (away from your body) and tighten it and close the Velcro fastener.
- Place the cuff around a bare arm 1-2cm above the elbow joint.
- While seated, place palm upside in front of you on a flat surface such as a desk or table. Position the air tube in the middle of your arm in line with your middle finger.
- The cuff should fit comfortably, yet snugly around your arm. You should be able to insert one finger between your arm and the cuff.

6. Body posture during measurement

Sitting Comfortably Measurement

- Be seated with your feet flat on the floor, and don't cross your legs.
- Place palm upside in front of you on flat surface such as a desk or table.
- The middle of cuff should be at the level of the right atrium of the heart

Once done, press the button Start. The device will automatically straight your arm, keep calm and relax. Do not talk. Please wait until the screen displays final results. Please find below a template to analyze these results.

7. Measurement

Press the **START/STOP** button. Remain still and do not move your arm until the entire measurement process is completed.

Make sure your monitor is level with your heart.

To stop the inflation or measurement, push the **START/STOP** button.

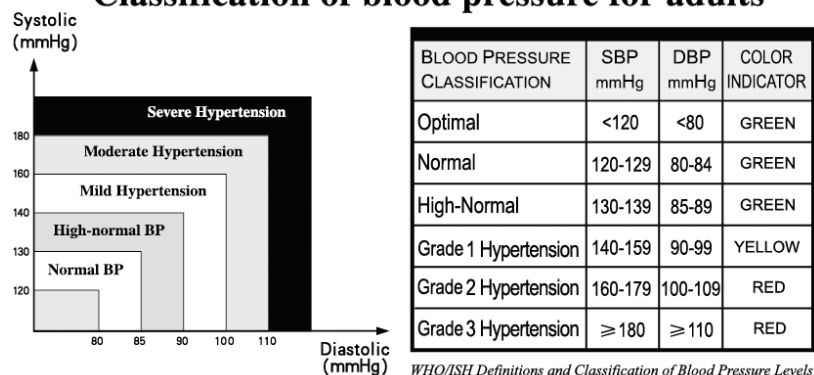
3. Inflation stops automatically and the measurement is started. As the cuff deflates, decreasing numbers appear on the display

a. Press the **START/STOP** button to turn the monitor off.

The monitor will automatically turn off after 2 minutes.

8. Display reading

Classification of blood pressure for adults

9. Using the memory function

The monitor is designed to store the blood pressure and the pulse rate about 90 sets in the memory.

1. Press the **MEM** button to display the values

2. Press the **MEM** button again when displayed average values, then will display the values from new to old stored in the memory.

3. Repeat press the **SET** button to circulate the measurements

TO DELETE ALL VALUES STORED IN THE MEMORY: Press the **MEM** button and the **SET** button together, when it shows “----” to delete the memory.

10. Trouble shooting tips

PROBLEM	CAUSES AND SOLUTION
No power. No display appears on the unit	Replace all batteries with new ones. Check the battery installation for proper placement of the battery polarities
Measurement values appear too high or too low.	Many factors including stress, time of day how you wrap the cuff, may affect your blood pressure. Take another measurement and remain still until the measurement is complete.
Still show and reading when press POWER without inflating	It may under the MEMORY mode. Stop the monitor and start another measurement
Blood pressure varies constantly.	Stress or anxious Stay clam and feel relax

11. Specifications

Measurement:	Oscillometric method
Measurement Range:	Pressure: 0 to 299 mmHg (0~39.9kPa) Pulse rate: 40 to 180 beats/min
Accuracy:	Pressure: Within ±3 mmHg Pulse rate: Within ±5% of reading
Inflation:	Automatic inflation with pumping
Air Release:	Automatic rapid air release by electromagnetic control valve
Storage capacity	90 sets measured values
Power supply:	4*1.5V AAA batteries, DC 6V
Battery life span:	100 times can be measured if using alkaline dry cell under 23°C in house
Operating Temperature and Humidity:	10°C to 40°C; 30 to 85% RH
Transportation and storage temperature & humidity:	10°C ~ 40°C; 30 ~ 85% RH
Accessories:	2*1.5V AAA Batteries packed, Instruction Manual, certificate of quality

Safety instructions

- The device is intended solely for domestic setting.
- If you have doubts medical, talk to your doctor before to use the monitor.
- Use the device only as directed in the manual. Any other use avoids the warranty rights.
- If you suffer from diseases, for example arterial occlusive disease, consult your doctor before use.
- The unit must not be used to control the frequency of a pacemaker.
- Pregnant women should take necessary precautions and take into account their personal limits. Talk with your doctor if necessary.
- If you feel being able discomfort, such as a sore arm, immediately deflate the cuff. Deflate cuff and remove it from the arm.
- This appliance is not intended for persons (including children) with physical, sensory or mental capabilities or lack of experience and / or knowledge unless they are supervised by a person responsible for their safety or have been investigated the use of the device.
- Children should be supervised by ensuring that they do not play with the device.
- The monitor should not be used near devices that emit a strong electric radiation, such as radio transmitters, mobile phones or microwave ovens.
- In case of malfunctions, do not repair the unit yourself, as this will invalidate all warranty rights. Talk to approved centers for repairs.
- Keep the device away from moisture. If moisture gets inside the unit, immediately remove the batteries and discontinue use of this device. Contact your dealer in this case or inform us directly.
- If you do not you serve and hold the camera, remove the batteries.

WARNING

Do not take therapeutic measures

Based on measurements taken by yourself.

Never change the doses of drugs prescribed by your doctor.

MAINTENANCE

1. Do not drop this monitor or subject it to strong impact.
2. Avoid high temperature and solarization. Do not immerse the monitor in water as this will result in damage to the monitor.
3. If this monitor is stored near freezing, allow it to acclimate to room temperature before use.

4. Do not attempt to disassemble this monitor.

5. If you do not use the monitor for a long time, please remove the batteries.

6. Clean the monitor with a dry, soft cloth or a soft cloth squeezed well after moistened with water, diluted disinfectant alcohol, or diluted detergent.

The European directive 2012/19/EC about electronic and electric waste, requires that you can't throw away defective domestic appliance with common waste. Used device has to be collect separately to optimize the recuperation rate and the recycling of the materials to reduce the impact on the health and the environment.

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capacities, or by persons lacking in experience or knowledge, unless they are supervised or

instructed on the use of this apparatus by a person responsible for their safety. Children should be supervised to ensure they do not play with this appliance.

Precision and precautions about the battery use

- Throwing away a used battery in the environment or with household waste pollutes and avoids the recuperation of reusable materials.
- It is important to limit the battery consumption and to respect the following rules:
 - Favor alkaline battery (last longer than saline battery) and when it's possible use rechargeable battery.
 - Leave used batteries and accumulators in adapted containers that you can find in some retailers. In that way, metals don't pollute the environment and the health.
 - Battery has to be inserted by respecting the polarity show on the apparel and the battery. An incorrect positioning can damage the apparel, cause leaking and even provoke fire and the explosion of the battery.
 - To assure a good functioning, battery has to be in a good condition. If there is a problem, put new battery.
 - Never try to recharge battery not made for this. It could leak, warm up, cause fire or explode.
 - Replace all the battery at the same time. Never mix up alkaline battery with saline battery.
 - Used battery has to be removed.
 - Also, remove the battery if you're not using the apparel for a long time.
 - Never try to short-circuit the terminal of a battery.
 - Never throw away the battery in fire because they can explode.
 - Take out the accumulator from the apparel before reload it.
 - We recommend that an adult watch if a child change the battery to respect those rules.
 - If you swallow a battery, go immediately to your doctor or the hospital.

Importé par / Importiert von / Geïmporteerd door / Imported by :
CAPTELEC, 59170 Croix - FRANCE