Artikel | Item | Article
Artikel | Articolo | Articulo

4509.1.00

4509GAz_1508

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ENGLISH

GB

1 Introduction	17
2 Important safety hints	18
2.1 Contraindication	18
2.2 Adverse Reactions	19
3 Graphic illustration	19
3.1 Front and rear panel	19
3.2 LCD Display	19
4 Operating instruction for battery and electrodes	20
4.1 Battery	20
4.1.1 Disposal of battery	20
4.2 Connected electrodes to lead wires	20
4.3 Connect Leadwires to device	20
4.4 Electrode	20
4.4.1 Place electrodes on skin	20
4.4.2 Electrode placement	21
5 Operating instructions	21
5.1 Insole pads	21
5.2 Turn on	21
5.3 Operation in the "Power off" mode	21
5.4 Operation in the "Standby" mode	21
5.5 Operation in the "output" mode	22
5.6 In all operations modes	22
6 Process chart and function illustration	22

7 Cleaning and Care	23
7.1 Tips for skin care	23
7.2 Cleaning the device	23
7.3 Electrodes	23
7.4 Cleaning the electrodes cord	23
7.5 Maintenance	23
8 Troubleshooting	24
9 Delivery Contents	24
10 Technical data	25
10.1 Technical specifications for TENS mode	25
10.2 Technical specifications for EMS mode	25
11 Explanation of symbols on unit	26
12 Disposal	26
13 Warranty	26

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1 | Introduction

Thank you for purchasing the TENS & EMS Dual pad. Please read this Instruction Manual thoroughly before using this unit or the safe and correct use of it. Please keep this Instruction Manual always at hand for your future reference.

The advantages:

- This is a quality product that should last a long time when used properly.
- It activates the body's own painkilling mechanisms, which sends low-intensity electrical pulses to the nerves below the skin relieving chronic pain in a range of cases. It has been backed by studies conducted over the world that have proved that tens for pain is very effective and has none of the side effects.
- The TENS device is very easy to use and very discreet.
- Its small size makes it convenient to wear all day and this allows you to reduce pain and return to an active lifestyle.
- The foot sole pads allow for easy stimulation of the foot bottom instead of using the normal pads.
- The stimulation of the foot soles serves to relieve swollen and painful feet.
- The stimulator is suitable for private use.

Medical background - Explanation of pain:

Pain is a warning signal -we need these signals to tell us that something may be wrong with our body. Without it, we may do not know that part of our body might be damaged, thereby damaging them further. However, once we have identified damage, pain serves little purpose. Pain does not begin until coded message travels to the brain where it is decoded, analyzed, and then reacted to. The pain message travels from the injured area along the small nerves leading to the spinal cord. Here the message is switched to different nerves that travel up the spinal cord to the brain.

The pain message is then interpreted, referred back and the pain is felt.

What is electrical stimulator and what can it do?

Digital TENS/EMS belongs to the group of electrical stimulation systems. It has two basic functions, which can be used in combination.

- 1. Electrical stimulation to nerve tracts (TENS)
- 2. Electrical stimulation to muscle tissue (EMS)

It has various versatile functions in order to increase general well-being, alleviate pain, maintain physical fitness, relaxation, and revitalize muscles and combat fatigue. The operation principle of electrical stimulation equipment is based on simulating the body's own pulses which are transmitted transcutaneous to nerve or muscle fibers by means of electrodes. The electrodes can thereby be attached to many parts of the body, whereby the electrical stimuli are safe and practically painless. You merely feel a gentle prickling or vibrating in some applications. The electrical pulses transmitted to the tissue affect the transmission of stimulation in nerve conductions as well as Neuron and muscle tissue in the field of application. Electrical stimulation does not replace regular exercising of the muscle, but is able to reasonably supplement the effect thereof.

What is TENS?

TENS, Transcutaneous Electrical Nerve Stimulation, means electrical stimulation of nerves through the skin. It is a non-medical method of treating pain from certain causes. TENS uses tiny electrical impulses sent through the skin to nerves to modify your pain perception. TENS does not cure any physiological problem; it only helps control the pain.

TENS does not work for everyone; however, in most patients it is effective in reducing or eliminating the pain, allowing for a return to normal activity.

TENS is clinically tested and approved with the following applications:

- Back pain, particularly also lumbar and cervical spine problem
- Headaches
- Joint pain (e. g. knee joint, hip joint, shoulder)
- Pain after injuries to musculoskeletal system
- women's period pains, dysmenorrhoea
- Pain with circulatory problems
- Neuralgia
- Chronic pain through various causes

What is EMS?

EMS, Electrical Muscle Stimulation, is a widespread and generally recognized method and has been used for years in sports and rehabilitation medicine.

In the sports and fitness field, one of the uses of EMS is as a supplement to conventional muscle training, in order to increase the efficiency of muscle groups and adapt physical proportions to the desired aesthetic results. EMS application goes in two directions: On one hand, a targeted strengthening of musculature can be produced (Activating application) and on the other hand a relaxing, resting effect can also be achieved (relaxing application)

Active application includes the following:

- Muscle training to improve endurance performance and/or
- Muscle training to support the strengthening of certain muscles or muscle groups in order to achieve desired changes to body proportions.

Relaxing application includes the following:

- Muscle relaxation in order to loosen up muscular tension
- Improving muscular fatigue symptoms
- Accelerating muscle regeneration after high muscular performance (e.g. after a marathon)

2 | Important safety hints

Retain the manual for further use and forward it to other users!

- Use of the device does not replace medical consultation and treatment.
- In the event of any type of pain or illness, you must therefore always first ask your doctor.
- Read, understand and practice the precautionary and operating instructions.
- Be sure to comply with all "Warnings, Cautions and Adverse reactions" in the manual, in order to prevent any damage to health.

2.1 | Contraindication

Do not use in the following cases:

- Users with implanted electronic equipment, such as pacemakers and intracardiac defibrillators who have not gotten doctor's permission must not use the device
- Epilepsy
- Serious arterial circulatory problems in the lower limbs
- Abdominal or inguinal hernia
- If you have heart disease without consulting your physician
- Safety has not been established for the use of therapeutic electrical stimulator during pregnancy
- This device should not be used when cancerous lesions are present in the treatment area
- with high fever
- Please don't use it in the bathroom or other place with high humidity.
- Do not use after consuming alcohol
- Do not use the device if you are connected to, or in the vicinity of, high-frequency surgical or industrial equipment. This may cause burn injuries on the skin under the electrodes, as well as problems with the stimulator.
- Please do not place pads near heart, genitals or diseased skin.
- Suggestion: Do not use the device when close to short wave or microwave therapeutic instrument (such as 1M), as the use of low frequency instrument at this time is not stable in output.

**Caution!**

- Keep yourself informed of the contraindications.
- Stimulation should not be applied over throat or mouth. Severe spasm of the laryngeal and pharyngeal muscles may occur and the contractions may be strong enough to close the airway or cause difficulty in breathing.
- Stimulation should not be applied anywhere on the front of the thorax in that the introduction of electrical current into the heart may cause cardiac arrhythmias and lead to cardiac arrest.
- This stimulator not intended for unattended, personal use by patients who have noncompliant, emotionally disturbed, dementia, or low IQ.
- Do not use if you have pains of unknown origin
- Do not place electrodes on your head or at any sites (i. e. for head area electrode not allowed to be placed at front and back, or left and right in pairs) that may cause the electrical current to flow trans cerebrally (through the head)
- If the stimulator levels are uncomfortable or become uncomfortable, reduce the stimulator intensity to a comfortable level and contact your physician if problems persist.
- Never use the device in rooms where aerosols (sprays) are used or pure oxygen is being administered.
- Do not use it near any highly flammable substances, gases or explosives.
- Inspect Applicator cables and associated connectors before each use.
- Turn the device off before applying or removing electrodes.
- Electrical stimulators should be used only with the leads and electrodes recommended for use by the manufacturer.

- Before using the device, you should consult with the doctor in charge of your treatment, especially in the event of the following:
 - Acute diseases, especially if hypertension is suspected or actually exists, thrombus diseases or Malign bump
 - All skin diseases
 - Chronic pain disorders with reduced pain awareness (such as metabolic disturbances)
 - Medical treatments carried out simultaneously


2.2 | Adverse Reactions

- Possible skin irritation or electrode burn under the electrode pad may occur.
- If skin irritation occurs, discontinue use and consult your physician.
- If the stimulation levels are uncomfortable, reduce the stimulation Intensity to a comfortable level and contact your physician if problems persist.

3 | Graphic illustration


3.1 | Front and rear panel

- 1 Channel A output socket:
Output of Channel A after connection of the cable with adhesive electrodes
- 2 Channel B output socket:
Output of Channel B after connection of the cable with adhesive electrodes
- 3 LCD Display:
Shows various user parameters and information
- 4 Button [+]
To increase output intensity of Channel A
- 5 Button [⏻]
Power on/off
- 6 Button [-]
To decrease output intensity of Channel A
- 7 Plastic shell
- 8 Button [TENS]
Choosing TENS function
- 9 Button [+]
To increase output intensity of Channel B
- 10 Button [-]
To decrease output intensity of Channel B
- 11 Button [EMS]
Choosing EMS function
- 12 Typ Label
- 13 Battery cover
- 14 Hanging hole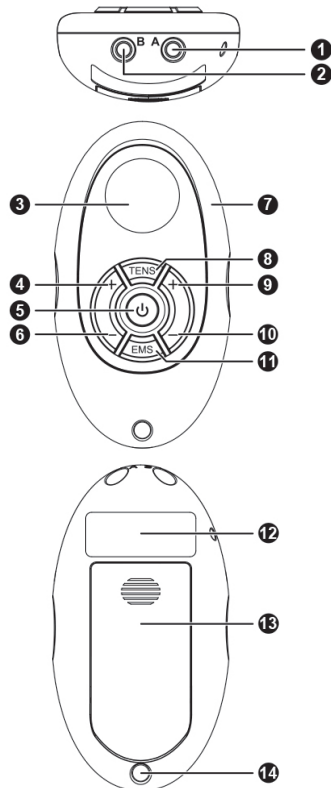

3.2 | LCD Display

Remaining time of application

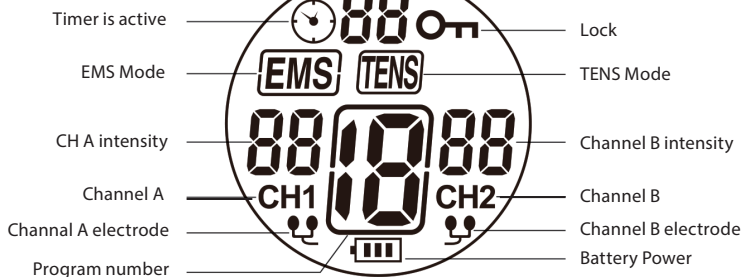4 | Operating instruction for battery and electrodes

4.1 | Battery

- a Slide the battery compartment cover **13** and open it.
- b While removing the battery, just slightly pick up the belt under the battery compartment.
- c Insert the 3 units of AAA 1.5V batteries into the battery compartment. Be sure to match the positive and negative ends of the battery to the marking in the battery compartment of the device.
- d After replacing the battery, make sure compartment cover to be tightly closed.

4.1.1 | Disposal of battery

- Spent batteries do not belong in the household waste.
- Dispose of the battery according to the current regulations.
- As a consumer, you are obligated by law to return spent battery to the Recycle Bin

⚠ Caution!

- If a battery has been swallowed please seek medical attention immediately!
- Increased risk of leakage, avoid contact with skin, eyes and mucus membranes. Rinse the affected spots with lots of clear water immediately and contact a physician right away.
- Battery may not be dismantled, thrown into fire or short-circuited.
- Protect battery from excess heat.
- Take the battery out of the product if they are spent or in case you no longer use the article. This prevents damage caused by leaking battery
- Replace all of the batteries simultaneously!
- Always replace the same type battery

4.2 | Connected electrodes to lead wires

Insert the lead wire connector into electrodes connector. Make sure they are properly connected to ensure the good performance. (Please refer to pic. 1)

Pic.1

- ⚠ Caution!** Always use the electrodes with the requirements of the IEC/EN60601-1 ,ISO10993-1/-5/-10 and IEC/ EN60601-1-2, such as with CE mark, or which are legally marketed in the US under 510(K) procedure.

4.3 | Connect Leadwires to device

Before proceeding to this step, be sure the device is completely turns OFF.

- a The wires provided with the system insert into the jack sockets located on top of the device. Holding the insulated portion of the connector, push the plug end of the wire into one of the jacks one or two sets of wires may be used.
- b This device has two output circuit at the top of the unit controlled by Channel A and Channel B. You may choose to use one channel with one pair of lead wires or both channels with two pairs of lead wires. Using both channels gives the user the advantage of stimulating two different areas at the same time.

- ⚠ Caution!** Do not insert the plug of the lead wire into any AC power supply socket.

4.4 | Electrode

- The electrodes are disposable and should be routinely replaced when they start to lose their adhesive nature. If you are unsure of your electrode adhesive properties, please order new replacement electrodes.
- Replacement electrodes should be re-ordered through or on the advice of your physician the manufacturer of the device to ensure proper quality.
- Follow application procedures outlined in electrode packing, to maintain optimal stimulation and to prevent skin irritation.

4.4.1 | Place electrodes on skin

Apply electrodes to the exact site indicated by your physician or therapist. Before applying electrodes, be sure the skin surface over which electrodes are placed is thoroughly cleaned and dried. Make sure the electrodes are placed firmly to the skin and make good contact between the skin and the electrodes.

- Inspect your electrodes before every use. Replace electrodes as needed. Reusable electrodes may cause slight skin irritation, lose adhesion and deliver less stimulation if overused.

To use the electrodes:

- Attach the electrode to the lead wire.
- Remove the protective backing from the electrode surface. Do not throw away the protective backing because it is reused after the treatment session has been completed.
- Place the tacky surface to the prescribed skin area by pressing the electrode firmly against the skin.

To remove your electrodes:

- Lift the corner of the electrode and gently remove it from the skin.
- Apply the protective backing to the tacky side of the electrode.

**Caution!**

- Before applying the self-adhesive electrodes, it is recommended to wash and degrease the skin, and then dry it.
- Do not turn on the device when the self-adhesive electrodes are not positioned on the body.
- To remove or move the electrodes, first switch off the device or the appropriate channel in order to avoid unwanted irritation.
- It is recommended that, at minimum, 4 cm x 4 cm self-adhering based, square electrodes are used at the treatment area.
- Do not apply to broken skin.
- The electrodes should be discarded when they are no longer adhering.
- The electrodes are intended for single patient use only.
- If irritation occurs, discontinue use and consult your clinician.
- Always remove the electrodes from the skin with a moderate pull in order to avoid injury in the event of highly sensitive skin.
- Do not pull on the electrode wire. Doing so may damage the wire and electrode.

4.4.2 | Electrode placement

- The placement of electrodes can be one of the most important parameters in achieving success with therapy.
- Every patient is differing from the responds to electrical stimulation, and their needs may vary from the conventional settings suggested here.
- Once an acceptable placement has been achieved, mark down the electrodes sites and the settings, so the patient can easily continue treatment at home.


5 | Operating instructions

5.1 | Insole pads


- Prepare all kinds of accessories. (as pic. 2)
- Put your right foot on the front of the insole. Get the White belt to pass through the bottom of insole, and bond the belt on the instep (the degree of tightness is adjustable). Left foot follow this. (as pic. 3)
- Insert the electrode wire to the plug fetched out from the insole. Please match the color of the plug to that of the receptacle. (as pic. 4)
- Connect the electrode wire to host. (as pic. 5)

Pic 2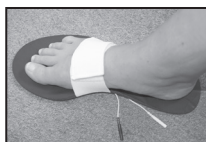

Pic 3

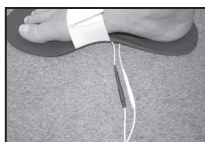Pic 4

Pic 5

5.2 | Turn on

- Before using the device for the first time, you are strongly advised to take careful note of the counter-indications and safety measures detailed at the beginning of this manual (Safety information). As this powerful equipment is a powerful equipment not a toy!

5.3 | Operation in the “Power off” mode

- There's no display in the LCD under “Power off” mode.
- In order to turn on the device, device start up into standby mode.

5.4 | Operation in the “Standby” mode

- There are 2 types of therapeutic mode in stimulator:

- TENS (Transcutaneous electrical nerve stimulation), and EMS (Electrical muscle stimulation).
- Under this mode press [TENS] button to choose TENS mode.
- Under this mode presses [+] button enter into treatment mode.
- Under TENS function press [EMS] button enter into EMS function. LCD shows EMS program 1.
- Under EMS function press [EMS] button to adjust program No.
- Under EMS function press [TENS] button enter into TENS function, LCD shows TENS program 1.
- Under TENS function press [TENS] button to adjust program No.
- 5 TENS programs as well as 5 EMS programs are available.
- Under this mode press [⏻] button return to "Power off" mode.

5.5 | Operation in the "output" mode

- Under this mode press [⏻], output wave shape stops and power off.
- Under this mode press [+], the output intensity of channel A will increase 1 level, and amplitude output appears 1 increase.
- Under this mode press [-] to decrease intensity, wave amplitude decreased and transferred to 0 to return to standby mode.
- Under this mode, if electrodes pad are not stacked well, after two pulse cycles will automatically return to standby mode and corresponding electrode symbol will show up.
- Under this mode, to press [EMS] under TENS function will enter into EMS function, LCD display EMS 1 and intensity at the lowest intensity 0 output.
- Under this mode, to press [TENS] under EMS function will enter into TENS function, LCD display TENS 1 and intensity at the lowest intensity 0 output.
- Output will stop after the end of each countdown and return to standby mode. The remaining time of operation is shown in the display.
- Note: When turn on the device; the output mode is the last treatment mode even after the batteries have been removed.

5.6 | In all operations modes

- Power supply voltage less than Approx. 3.3V the battery symbol will blink and remind the user to change the battery. In this status the device will turn off after 180 seconds.

Caution: If the stimulation levels are uncomfortable or become uncomfortable, reduce the stimulation intensity to a comfortable level and contact your medical practitioner if problems persist.

6 | Process chart and function illustration

TENS Program table:

Program	Workingtime (min)	Frequency	Pulse width (μS)	Waveform property
TENS 1	30	80	200	normal
TENS 2	30	2	250	normal
TENS 3	30	2	200	normal
TENS 4	30	2/80	250/200	han
TENS 5	30	2~80	200	FM

EMS Program table:

Program	Workingtime (min)	Frequency	Pulse width (μS)	Waveform property
EMS 1	30	50	250	EMS
EMS 2	30	20	250	EMS
EMS 3	20	30~80	250	EMS
EMS 4	25	30~45	250	EMS
EMS 5	30	30~45	250	EMS

7 | Cleaning and Care

7.1 | Tips for skin care

To avoid skin irritation, especially if you have sensitive skin, follow these suggestions:

- Wash the area of skin where you will be placing the electrodes, using mild soap and water before applying electrodes.
- Be sure to rinse soap off thoroughly and dry skin well
- Excess hair may be clipped with scissors; do not shave stimulation area.
- Wipe the area with the skin preparation your clinician has recommended. Let this dry. Apply electrodes as directed.
- Many skin problems arise from the “pulling stress” from adhesive patches that are excessively stretched across the skin during application. To prevent this, apply electrodes from centre outward; avoid stretching over the skin.
- To minimize “pulling stress”, tape extra lengths of lead wires to the skin in a loop to prevent tugging on electrodes.
- When removing electrodes, always remove by pulling in the direction of hair growth.
- It may be helpful to rub skin lotion on electrode placement area when not wearing electrodes.
- Never apply electrodes over irritated or broken skin.

7.2 | Cleaning the device

- Remove the battery from the device every time when you clean.
- Clean the device after use with a soft, slight moistened cloth.
- In case of more extreme soiling you can also moisten the cloth with mild soapy water.
- Ensure that no water penetrates into the device.
- Do not use any chemical cleaners or abrasive agents for cleaning.

7.3 | Electrodes

- It may be helpful to improve repeated application by spreading a few drops of cold water over the adhesive and turn the surface up to air dry. Over Saturation with water will reduce the adhesive properties.
- Between uses, store the electrodes in the resealable bag in a cool dry place.

7.4 | Cleaning the electrodes cord

- Clean the electrode cords by wiping them with damp cloth.

7.5 | Maintenance

- Maintenance and all repairs should only be carried out by an authorized agency.
- The manufacturer will not be held responsible for the results of maintenance or repairs by unauthorized persons.
- The user must not attempt any repairs to the device or any of its accessories.
- Please contact the retailer for repair.
- Opening of the equipment by unauthorized agencies is not allowed and will terminate any claim to warranty.
- Check the unit before each use for signs of wear and/or damage.
- Replace wear items as required.

8 | Troubleshooting

If your device does not seem to be operating correctly, refer to the chart below to determine what may be wrong. Should none of these measures correct the problem, the device should be serviced.

Problem	Case	Remedial Action
The power cannot be turned on	The batteries are inserted in wrong direction	Insert the batteries in correct direction
The output intensity turns off	1) The pads are not proper attached to the skin	1) If the pads are not attached proper to the skin, the output intensity will turn off automatically
	2) Pads are removed from the skin	2) If the pads come off from the skin, the power will turn off automatically
	3) The batteries are worn out	3) Replace the batteries with new ones

	4) The electrodes cord is broken	4) Replace the Electrode leads
The PADs does not attach to the skin.	The adhesive surface of pad is dirty	Pad is a consumable. Replace pad with a new one
Cannot feel the stimulation	1) The pad is not attached close to the skin	1) Attach the pad close to the skin.
	2) The pads are attached overlapping each other.	2) Attach the pads by not overlapping them.
	3) The Electrode leads is not connected correctly.	3) Connect the Electrode leads correctly.
	4) The intensity is adjusted too weak.	4) Turn up the Intensity with the [+] Dial/Power Button.
	5) The batteries are worn out.	5) Replace the batteries with new ones.
	6) The adhesive surface of Pad is damaged.	6) Pad is a consumable. Replace the pad with a new one.
The skin turns red. The skin feels irritating.	1) The treatment time is too long.	1) Limit the treatment time to 20 to 30 minutes.
	2) The pad is not attached close to the skin.	2) Attach the Pad close to the skin.
	3) The adhesive surface of pad is damaged.	3) Pad is a consumable. Replace the pad with a new one

9 | Delivery Contents

- Electrical Stimulator Device, 1 piece
- Electrode pad (50mm x50mm), 4 pieces
- Electrode leads with 2 buttons, 2 pieces
- Y-cable for footsoles with (4 plugs), 1 piece
- Insole, 2 pieces
- White belt (30cm x 5cm), 2 pieces
- AAA batteries, 3 pieces
- Instruction Manual, 1 piece

10 | Technical data

Model:	FDES108
Working voltage:	DC 4,5 V
Battery type:	3 x AAA
Channel:	Dual isolated between output channels
Maximum Amplitude:	40 V
Load Impedance:	500 Ohm
Power save time:	appr. 3 minutes
Working current:	appr. 150 mA
Dimensions:	110 x 59.5 x 30 mm
Weight:	appr. 45 gr without Battery
Tolerance:	20% of output intensity
Electrode detection function:	The amplitude level will be reset to 0 when the amplitude level is 1 or greater and an open circuit at either Channel is detected.
Operating conditions:	a temperatur range from +5°C to +40°C a relative humidity range of 15% to 93% non condensing an atmospheric pressure range of 700hPa to 1060 hPa
Storage conditions:	-25°C without relative humidity control; and to 70°C at a relative humidity uo to 93% non condensing

10.1 | Technical specifications for TENS mode

Waveform	single alternate square pulse wave
Pulse width:	from 200 to 250 μ S depends on the different programs
Pulse rate:	from 2 to 80Hz according to different programs
Burst:	TENS 3: There is one pulse wave includes 7 pulses which is 100 Hz per 2 Hz output.
Normal:	TENS 1: The pulse rate and pulse width output will be 80 Hz / 200 μ S; TENS 2: The pulse rate and pulse width output will be 2 Hz / 250 μ S;
Pulse width modulation:	TENS 4: The pulse width is automatically varied in a cycle time. The pulse width output is 2Hz/250 μ S in the first 3 second and 80 Hz/200 μ S in another 3 seconds. Thus run in cycles.
Pulse rate modulation:	TENS 5: The pulse rate is automatically varied in a cycle time. The pulse rate is increased from 2 Hz to 80 Hz in setting cycle time 5 s, keeps 1second and then decreased from 80 Hz to 1 Hz in nest setting cycle time 5 s and keep 1 s, thus run in cycles. In this program, pulse rate (2 to 80 Hz), pulse width 200 μ S

10.2 | Technical specifications for EMS mode

Pulse Width:	250 microseconds
Puls Rate	EMS 1: 50 Hz EMS 2: 20 Hz EMS 3: 30~80 Hz EMS 4: 30~45 Hz EMS 5: 30~45 Hz
Ramp up time:	Output amplitude rising edge time within a period 2s \pm 30%
On time:	Output amplitude keeping time within a period 5 \pm 30%
Ramp down time:	Output amplitude trailing edge time within a period 2s \pm 30%
Off time:	Output amplitude zero output time within a period 10s \pm 30%

11 | Explanation of symbols on unit

	LOT		Not suitable for children under 3 years!
	SN		Waste electrical products should not be disposed of with household waste
IP22	IP22		Symbol for MANUFACTURER
	Symbol for CONSULT Operation Guide		COMPILES WITH MDD93/42/EEC REQUIREMENTS
	Symbol for TYPE BF APPLIED PARTS		

12 | Disposal

Electrical and electronic devices may not be disposed of with domestic waste. Consumers are obliged by law to return electrical and electronic devices at the end of their service lives to the public collecting points set up for this purpose or point of sale. Details to this are defined by the national law of the respective country. This symbol on the product, the instruction manual or the package indicates that a product is subject to these regulations. By recycling, reusing the materials or other forms of utilising old devices, you are making an important contribution to protecting our environment.

13 | Warranty

The guarantee lasts for 24 months from the date of purchase for manufacturers' error in material and quality. If it is used for another purpose or operated incorrectly, no liability for possible damages can be accepted. This guarantee is only valid if the instructions in the "instructions for use" have been followed. Warranty is void if product is misused, or repairs attempted by unqualified persons.

Dear Customer.

In case you product does not function as you expected please do not return immediately. Perhaps our Hotline (+49-69-954061-24) can be of assistance. If you are still not satisfied, you have the possibility to return products free of charge within the BRD. We require your address. This you may provide to the above listed telephonic hotline or by email to: technik@hydas.de. Prepare the package for shipping and include the receipt and description of the problem. Please be aware of the fact that collect packages may not be accepted.

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Hydas GmbH & Co. KG
c/o atrikom fulfillment GmbH • Haagweg 12
Hydas GmbH & Co.KG
c/o atrikom fulfillment GmbH
Hydas Inc.
Hummelstown, PA. 17036 U.S.A.