# Sound Genie®

**Deutsch**

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Sound  Genie®

# i

Sound Genie is a sound amplifier and can be used as a hearing aid alternative. Please keep these instructions in a safe place for future reference. Due to continuous product improvement, the picture on the packaging may differ slightly from the actual product.

Please retain Instructions for future reference.

## RECHARGEABLE SOUND GENIE

Rechargeable Sound Genie is a sound amplifier that helps amplify sound up to 30X, with no worries of having to replace batteries! It is equipped with a rapid charger and a rechargeable battery that can last up to 8 -10 hours of continuous use. It includes 6 interchangeable ear tips to SEAL OUT the noise!

Set includes:

- Rechargeable Sound Genie Earpiece
- Rechargeable Sound Genie Adapter
- Charging Docking Station
- Cleaning Brush
- Six (6) Silicone Adjustable Tips

Cleaning Brush

FIGURE A


Rechargeable Sound Genie Adapter

FIGURE B

Rechargeable SoundGenie Adapter  
Red Light "ON" when Chargingg

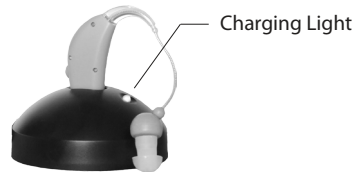FIGURE C


Sound Genie Charging Dock Station

FIGURE D

Six (6) Silicone Adjustable Tips

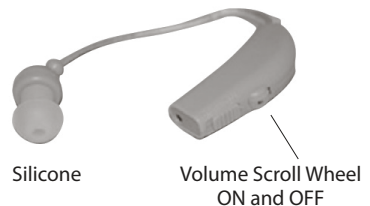FIGURE E

Silicone

Volume Scroll Wheel  
ON and OFF

FIGURE F

## CAUTION

- Rechargeable Sound Genie is NOT for use by children under 15.
- Keep out of reach of children and pets.
- The supply terminals are not to be short-circuited.
- Do NOT wet.

**Note:** Discontinue use of product in the event of irritation. It is recommended to consult a doctor before continued use of this product.

## GETTING STARTED

It is important to follow the instructions (1-3) below for the first two times charging the Rechargeable Sound Genie Sound Amplifier:

1. Slide the switch down located on the back of the earpiece to turn the Rechargeable Sound Genie OFF.
2. Connect the adapter's USB plug into the rechargeable docking station. Plug into an outlet and upon plugging, the red light on the adapter (See Figure C) will illuminate, and the green light on the docking station (See Figure D) will light simultaneously.
3. Place the Rechargeable Sound Genie into the recharging docking station. The green light on the docking station will turn red if connected properly.

**Note:** DO NOT REMOVE THE RECHARGEABLE Sound Genie FROM THE DOCKING STATION UNTIL THE LIGHT TURNS GREEN (AGAIN) - SIGNALING FULLY CHARGED.

**Important:** Follow these steps to ensure longer life.

## HOW TO WEAR THE RECHARGEABLE SOUND GENIE SOUND AMPLIFIER

Note: One (1) silicone ear tip is already on the Rechargeable Sound Genie Sound Amplifier.

- The Rechargeable Sound Genie includes 6 different silicone ear tips. Select the silicone tip that best fit the ear and place onto the Rechargeable Sound Genie Sound Amplifier.
- The Rechargeable Sound Genie can be worn on either the right or left ear. On the ear piece, make sure the volume tab is in position "1" before use.
- Below the volume scroll wheel is the ON/OFF switch. This must be in the "ON" position or in the "UP" position for use, end slide down for "OFF" to turn the device OFF.
- Take the ear piece and place it over the top of the desired ear. The ear piece will curve around and fit behind the ear.
- Carefully insert the silicone tip of the Rechargeable Sound Genie which is at the end of the small wire into the ear canal.
- Adjust the volume to the desired level.

**Note:** THE DEVICE WHISTLES IF THE VOLUME LEVEL IS TOO HIGH. ADJUST VOLUME ACCORDING TO COMFORT LEVEL.

**Note:** CHARGING TIME WILL VARY DEPENDING UPON THE LEVEL OF BATTERY POWER REMAINING PRIOR TO RE-CHARGE.

## GENERAL CARE

---

- Clean Rechargeable Sound Genie daily with the provided brush to avoid unwanted earwax build-up.
- Brush the device back-and-forth across the ear bud (using the supplied brush) to remove wax and dirt deposited within the ear bud.
- Make sure the ear bud is clear of all debris and dirt.
- Wipe the Rechargeable Sound Genie with a clean, dry cloth only. Do not wet.

**Note:** There are no serviceable parts to this item.

## STORAGE

---

When Rechargeable Sound Genie is not in use, clean it and store in a dry cool place.

## TROUBLESHOOTING

---

**IMPORTANT INFORMATION:** It is recommended consulting a certified physician before deciding to wear Rechargeable Sound Genie end to follow up with constant checkups to ensure none of the listed conditions below apply.

The following are conditions under which one should NOT wear the Rechargeable Sound Genie Sound Amplifier. Do not wear the Rechargeable Sound Genie if:

- Have pain and discomfort in the ear.
- Have traumatic or visible congenital deformities on your ear.
- Have acute or chronic dizziness.
- Have history of sudden or rapidly progressive hearing loss that has occurred within the last 90 days.
- Have unilateral hearing loss or sudden/recent onset within the last 90 days.
- Have significant build-up of wax or other materials in your ear canal.
- Bone conduction thresholds are better than the air conduction thresholds by 15dB or greater.

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HSP Hanse Shopping GmbH