# MAGICVISION

## LED-LESEBRILLE

„Eine für Alle“

**Deutsch**

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**English**

**Instruction Manual**  
from page 8

**Français**

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Dear customer,

we are pleased that you have decided to purchase this product. Should you have any questions, contact the customer service via our e-mail address:

**kundenservice@hsphanseshopping.de**

Please read the instructions carefully before using the device for the first time and keep them in a safe place. When passing on the appliance, please also hand over these instructions. The manufacturer and importer accept no liability if the information in this manual is not observed!

In the context of continuous development, we reserve the right to product, to change the packaging or package inserts at any time.

All safety instructions are marked with this symbol. Read them carefully and follow the safety instructions to avoid personal injury and damage to property.

## INTENDED USE

---

Thank you for purchasing the Magic Vision LED reading glasses. The fashionable reading glasses with LED reading light adjust to different visual acuities as needed. Read small print in package inserts, ingredient lists, instruction manuals, newspapers, on your mobile phone or computer without changing your glasses. Can help prevent eye fatigue, vision problems and headaches. The LED reading glasses guarantee excellent optical clarity. Rechargeable with included USB cable.

## APPLICATION

---

You can wear the Magic Vision LED reading glasses directly like ordinary glasses.

## CLEANING AND CARE

---

To remove light dirt and fingerprints, use the cleaning cloth (supplied). For very dirty lenses, use water or a neutral detergent to clean them. DO NOT use strong detergents as they may scratch the lenses.

## WARNINGS

---

Please note that the LED reading glasses are only suitable for near vision. The glasses are not suitable for driving or operating machinery. The LED reading glasses are not a substitute for everyday glasses. If you have any questions, please consult an optician or ophthalmologist.

## LED FUNCTION

---

These glasses are equipped with two LEDs. Through these LEDs an optimal illumination is guaranteed. The LEDs can be switched on and off individually via the switch. The supplied USB cable can be used to charge the permanently installed LI-ION batteries. The cable is designed in such a way that both batteries can be charged simultaneously. Connect the USB cable to a suitable adapter and then connect one or both temples of the LED reading glasses with the cable. The charging time is approximately four hours. The charging time depends on the charge level of the battery and the voltage used. A charge status of the glasses is not displayed. The optimum lighting duration of the LEDs is approx. eight hours. If you notice that the LEDs become weaker, recharge the glasses.

## SCOPE OF SUPPLY

---

1x glasses, 1x USB charging cable, 1x glasses cleaning cloth, 1x glasses strap, 1x glasses bag (pouch)

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HSP Hanse Shopping GmbH