Rollstuhl Actimo

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DE

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User Instructions

UK

Carefully read the user instructions before assembling, adjusting or using the Actimo Wheelchair for the first time. Keep the user instructions in a safe place so that they are available for later use. The user instructions contain all the information necessary for the adjustment and operation of the Actimo Wheelchair. If you have any questions or require further information, please contact your specialist dealer.

Intended Purpose

The Actimo Wheelchair is designed for indoor and outdoor use and is intended exclusively for the self-transport or assisted transport of people. Suitable for patients who, for example, cannot walk due to paralysis, physical weakness or cardiovascular disease and therefore need assistance with movement. The companion moves the wheelchair by pushing on the push handles provided for this.

Indications

The Actimo Wheelchair can be used in the event of restricted mobility of the legs and knee joints and the upper limbs, loss of strength, balance and slight visual impairment, inability to walk and / or a severe walking impediment

- Paralysis
- Loss of limbs
- Malfunction or deformity of the limbs
- Joint damage or joint contractures (not on both arms)
- Other illnesses

Instructions / Assembly: Have this carried out by your specialist dealer / care assistant

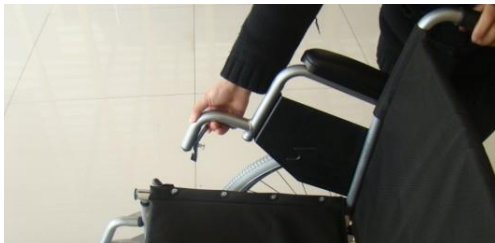
1. Remove all the individual parts from the cardboard box and check the contents and the packaging for damage and completeness.
2. The wheelchair is delivered pre-assembled. Hold the wheelchair firmly by the handles and slowly fold it apart.
3. To fold out the wheelchair, using both hands press down on the sides of the seat area until it locks into place. Take care not to hold the frame to avoid injury to the fingers.
4. Hang the mounts on which the foot flaps are mounted in the frame on the right and left of the wheelchair and turn them inwards, they audibly click into place. Fold the foot flaps down and adjust the height, to do so, loosen the locking nuts on both sides and pull out the mount to the desired length before re-tightening the locking nuts. Additional support is provided by the belt on the mounts.

To remove the mounts, operate the locking mechanism, swivel the mounts outwards and fold the foot flaps up.

Now the mounts can be pulled off upwards.

5. Adjustment of the seat height: To adjust this, loosen the front wheels in the forks and the axle screws of the rear wheels. Set the front and rear wheels to the desired height.

6. To loosen and fold up the armrests, press the locking lever at the lower end of the side frame. To fix the armrests, fold them down until the mount clicks into place (clearly audible).

7. When stationary, apply the parking brake (tyre brake). Press the lever for this forwards until it holds by itself. The wheelchair is now standing safely and you can now turn or simply just park. The parking brake is released by pulling the lever backwards.

8. To remove the rear wheels (drive wheels), press the button on the quick-release axle and grasp the spokes with four fingers. Then pull the drive wheel out of the axle mounting.

To fit the drive wheel, with the thumb press the button of the quick-release axle again. Now insert the drive wheel (the quick-release axle) in its mounting, the quick-release axle locks into place automatically.

9. The tilting aids make it easier for the companion to tilt the wheelchair and so comfortably move over obstacles. First push the anti-tilt wheels into the highest position. Place your foot on the lever and keep your balance using the push handles.

Contraindications (when using without external help)

The use of the Actimo Wheelchair is unsuitable in the event of:

- Perceptual disorders
- Severe balance disorders
- Loss of limbs on both arms
- Joint damage or joint contractures on both arms
- Inability to sit
- Reduced or inadequate eyesight

Safety Instructions and Setting of the Brakes

1. When unfolding the wheelchair take care not to grasp the the frame of the seat – there is the risk of trapping the fingers!
2. The **setting of the brakes** should be carried out by trained skilled personnel, however, if you have sufficient experience you can carry this out yourself.
With the brake mechanism open to the maximum, the clearance between the brake bolts and the tyre casing at the narrowest point should be 12-14 mm.
For the setting / correction of the clearance, loosen the screws. Then push the brakes into the correct position. Then re-tighten the screws and check the function of the brakes. The wheelchair should not be able to be pushed with the brakes applied.
With changes of the wheelbase, it may be necessary to completely remove the screws and fit the brakes in a slot further to the front or the back.
Attention! Incorrectly set brakes can have life-threatening consequences.
3. In order to avoid crashes and hazards, you should first practice the handling of the wheelchair on a level, open area. A companion should help you with this.
4. Never leave children and young people alone in an unsupervised wheelchair!
5. Before using the wheelchair, check that all fitted parts are correctly secured.
6. Before the first use, practice riding in the wheelchair on a level, open area. Make yourself thoroughly conversant with the braking and accelerating behaviour when travelling in straight lines and in curves.
7. Upholstered sections exposed to direct sunlight can be become hot and under certain circumstances can cause skin injuries on contact. Where applicable, cover these or protect the wheelchair against sunlight.
8. The wheelchair must not be misused for the conveyance of several people or loads. Observe the intended purpose!
9. Never use the wheelchair when under the influence of alcohol or other things which adversely affect awareness and / or physical and mental capacity!
10. Be aware that the shifting of balance through bodily movement or the loading of the wheelchair can increase the risk of tipping over.
11. Wherever possible please use access ramps for the overcoming of obstacles such as, for example, steps.
12. Wheel ruts, tram and train tracks, when they cannot be bypassed, should be crossed as straight (at right angles) as possible.
13. Above a gradient of 10° there is the risk of the wheelchair tipping backwards!
14. Avoid unrestrained pushing against an obstacle (step, kerbstone), the passenger could fall forwards!
15. Never stand on the footrests when getting in or out of the wheelchair!
16. Observe the road traffic regulations and, where possible, wear bright clothing in order to be better seen by other road users.
17. The passive lighting on the wheelchair must always be visible to other road users.
18. The maximum load-carrying capacity is 120 kg!

Maintenance / Maintenance Instructions for the Specialist Dealer

We recommend that the complete wheelchair is inspected by a specialist dealer at least once per year. In the event of malfunctions or defects on the wheelchair it must be immediately given to the specialist dealer for repair. We recommend that the product be treated in a correct and proper manner and that it be maintained in accordance with the instructions. Rehaforum Medical GmbH accepts no liability whatsoever for damage caused by components and spare parts which have not been approved by Rehaforum Medical GmbH. Repairs must be carried out by specialist dealers only.

If you notice unusual brake behaviour, inform your specialist dealer immediately and take the wheelchair out of use.

Do not use a defective wheelchair! ⚠

Squeaking sounds point to damage and should be investigated by a specialist dealer.

Independent Travel and Braking

Carry out your first trial journeys carefully in an even area with the aid of an assistant until you have accustomed yourself to your new wheelchair. Only following a lot of practice may you join in public road traffic. You must be especially careful when ascending or descending slopes or overcoming obstacles.

The wheelchair is moved and braked via the hand rim, in doing so, the hand is placed on the hand rim whereby only the thumbs and the bent index fingers are in contact with the hand rim. The other fingers are made into a fist and do not come into contact with the hand rim.

To brake, the hand rim is allowed to slip between the thumbs and the bent index fingers and braked to a standstill by a steadily tightening grip.

⚠ Attention: When propelling the wheelchair ensure that you do not touch the tyres with the thumbs. Risk of trapping between the tyres and the side pieces / armrests!

To steer the wheelchair, the hand rim on the side to which one wishes to steer or turn is braked.

To turn on the spot, one hand pushes forward and the other hand pushes backward at the same time.

Getting In and Out - Sideways

The wheelchair provides you with the possibility of getting in and out in a simple way. The footrests, footplates and the armrests can be folded up and / or swivelled. Bring the wheelchair and the present and future seats as close together as is possible.

By operating the right and left-hand brake levers, secure the wheelchair against unintended rolling way and swivel the armrests upwards and the footrests outwards with the foot plates folded up (where necessary, on both wheelchairs if you wish to move from one wheelchair into another one).

Now slide sideways onto the seat and in doing so ensure that you are sitting as far back as possible on the seat. Then return the armrests and footrests to their original positions.

Getting In and Out - from the Front

Fold up the foot plates and swivel the footrests outwards out of the way.

Bring the wheelchair and future seat as close as possible to each other head-on and secure the wheels via the parking brake. By turning the body, slide onto the seat and in doing so ensure that you are sitting as far back as possible. Then return the armrests and footrests to their original positions and fold the foot plates back down.

Checking of the Wheelchair (before each use)

- Integrity and adequate air pressure of the tyres (with optional pneumatic tyres). Once per month, regularly check the tread depth of the tyres, this should be at least one millimetre. Defective or worn tyres must be replaced.

- Functional efficiency of the brake system. With the brakes applied it should not be possible to move the wheelchair. With the brakes released, the wheelchair should be easy to move. If you notice unusual brake behaviour, inform your specialist dealer immediately and take the wheelchair out of use.
- Functional efficiency of the passive lighting system. Your wheelchair is equipped with yellow reflectors on the sides. The reflectors must always be visible to other road users and should not be dirty or obscured in any other way.
- Integrity and cleanliness of the seat and back upholstery. If these are dirty they can be cleaned in accordance with the cleaning instructions, if they are damaged, contact your specialist dealer.
- Moveable items and screw and rivet connections: Check these for ease of operation and integrity.

Instructions Regarding Transport

For transportation in a passenger vehicle or simply just to save space, the wheelchair can be easily folded together. In addition the drive wheels can be removed quite simply (see assembly description).

Attention: The wheelchair is not approved for use as a seat in vehicles such as disabled transport vehicles! Get out of the wheelchair and fold up the foot plates, where applicable, remove the leg supports and slowly pull the seat upwards, the Actimo Wheelchair then folds together.

Cleaning

Clean the Actimo Wheelchair regularly with a mild soap solution and clean water. Use a soft cloth for drying. Do not use any abrasive or corrosive cleaning agents.

The wheels can be cleaned using a damp brush with plastic bristles. Do not use wire brushes! (These damage the material!) The upholstery (cushions) can be gently (40°C) washed.

Use a commercially available disinfectant to disinfect the wheelchair. Do not use high-pressure or steam cleaners for cleaning!

Re-use / Multiple Use

On the transfer or re-use of the wheelchair it must be ensured that all the technical documents required for safe use are passed on to the new user. The Actimo Wheelchair is intended for daily use.

It might be used by several people. Before the wheelchair is used by someone else or ownership is transferred to another person, it must be thoroughly cleaned and disinfected in order to prevent the transmission of germs (contamination). Here the procedures described under the point *Cleaning* and a commercially available disinfectant must be used.

The wheelchair must be checked for damage and be approved by a specialist dealer!

Scope of Supply

Actimo Wheelchair, pre-assembled

Outer packaging (please keep for storage)

1 pair of foot supports

User instructions

Tool

Labelling and Type Plates

Type and manufacturer's data plates are fitted to the Actimo Wheelchair. The plates must always remain legible, illegible or missing plates must be replaced immediately. The type plate is very important for the indication of the product. It must not be removed.

Storage

Before being stored, the Actimo Wheelchair must be thoroughly cleaned and dried. Store the wheelchair in the outer packaging (cardboard box / polybag) in a dry location away from light.

Disposal

A worn-out transport wheelchair is a source of valuable materials which can be reintroduced into the industrial cycle. Therefore do not dispose of your transport wheelchair in the household waste, instead have it disposed of professionally by an officially approved waste disposal centre. Your local waste collection department will be pleased to provide you with the relevant information.

Guarantee

The guarantee period is 24 months and relates to all product defects which can be verifiably shown to be attributable to manufacturing or material defects

Technical Data / Dimensions and Weight

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