**Rollstuhlrampe RFM Aluminium**

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**Rollstuhlrampe RFM Aluminium**

Please read and retain this instruction, it provides essential information for correct and safe use. This instruction is guideline only.

**Introduction**

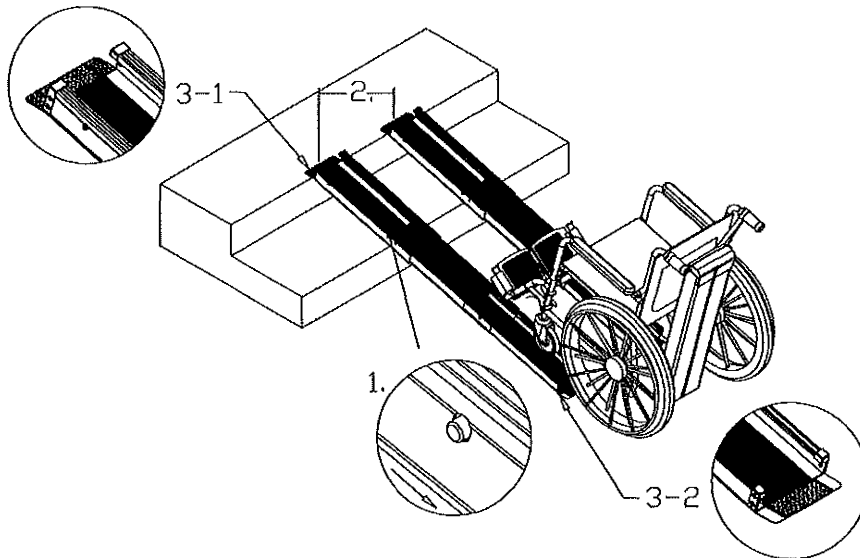
The portable aluminium ramps are made of aluminium, making them lightweight and easy to carry. With a black non-slip covering and a 40-50 mm edge to prevent the wheelchair from veering off, they are built safety in mind. Telescoping into three sections makes them compact and versatile.

**Specification**

Length	Internal Width	Weight	Material	Max. Load Weight
1,5 m	114 mm	5,5 kg	Aluminium	200 kg
2,1 m	114 mm	7,0 kg	Aluminium	200 kg

**Instructions for use**

1. Push in the end button and slide the section towards you until the catches pop into outside holes, double check the sections are locked securely.
2. Space the ramps apart so that the tires of the wheelchair are in the centre of the non skid ramp centres, to make this easier a length of cord is supplied, cut the cord to the same length as the distance between the wheelchair wheels and use it as a spacing guide at the top and the bottom of the ramps.
3. Make sure that the turned down tab is at the top of the ramp and the upturned tab is at the bottom.
4. Some surfaces may not be compatible with the non-skid under pads, make sure the ramps are securely positioned before using. If permanently positioned, it may be beneficial to screw the ramps in position.

### Instructions for Storing

1. Push the button in on the smaller section first and slide in until it stops.
2. Push in the second button on the largest section and slide in until it catches in the hole at the top of the ramp for storage.
3. Place the ramps in the carrying bag which has loops to hang on the back of the wheelchair.
4. To prevent the ramps from scratching each other, place a cardboard between the ramps.

### Warning

- Do not exceed the max. load of 200 kg. The weight of the user must be taken into consideration if the wheelchair or scooter is occupied.
- Recommend to use the ramps with the help of a qualified helper. The helper should not walk on the ramps.
- Always back the wheelchair down the ramps; never go in a forward direction.
- Ensure that the spring buttons are fully engaged in the holes.
- Before each use, make sure the product's safety and integrity.
- Periodically inspect for wear and damage.

### Maintenance

These ramps are made of anodized aluminium which gives them a tough tarnish resistant finish. The push buttons are made of zinc plated spring steel for years of rust free operation. To clean the ramps, use any mild detergent and water.

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